

Product Information NeoShades®

- ◆ **N720 NeoShades®** **Micro** **50/case**
- ◆ **N721 NeoShades®** **Small** **50/case**
- ◆ **N722 NeoShades®** **Large** **50/case**
- ◆ **Phototherapy Eye Shades with Cute Sunglass Design**

✓ **Features and Benefits:**

- ✓ **Attaches to patients with gentle NeoBond® hydrocolloid**
- ✓ Leaves no residue on skin
- ✓ No skin prep required
- ✓ No cumbersome straps around the head
- ✓ Available in three sizes
- ✓ Cute design encourages parents to photograph the baby
- ✓ Resists skin breakdown from moisture or heat
- ✓ Safe on neonatal skin
- ✓ Easy to apply and remove
- ✓ Latex-free
- ✓ Use in NICU, PICU, Respiratory, ER, Home Health

➤ **DFU**

- **Skin must be clean and dry per hospital protocol**
- It is important to have skin as clean and dry as possible for proper attachment
- Moisture, oils or lotions adversely affect the adhesion of tabs
- **DO NOT USE ALCOHOL**
- **APPLICATION:**
- Tabs are pre-attached to the NeoShades®
- Remove and reposition from NeoShades® if necessary for proper alignment on temples of patient
- Peel liner off the tabs and discard the liner
- Affix tabs to temples
- **Key: Hold in place for at least 60 seconds to ensure proper adhesion**
- Place NeoShades® over eyes and gently attach to tabs
- Leave a portion of the tabs exposed for easier removal
- **REMOVAL:**
- For feedings, bonding with parents or to check condition of eyes place fingers on edge of tabs and slowly peel NeoShades® off the tabs
- Extra tabs are enclosed in each case if needed
- **NOTE:** Once a tab has been removed from a patient, it cannot be re-applied
- **REMOVAL FROM PATIENT:**
- Use a cloth or swab with warm water and gently wipe the skin as you peel back the tabs
- Soap or mineral oil may also be used for removal
- **CAUTIONS:**
- Discontinue immediately if skin irritation occurs
- To be used by or on the order of a physician