

H😊ld-A-Line

Catheter & Tube Holder

Catalog# N705

Size: Standard

Packaging: 40/case

Directions For Use

1. Clean & Dry skin per hospital protocol
2. Remove liner from Hydrocolloid base
3. Apply base to patient & hold in place for 60 seconds
4. Remove liner from flap
5. Position tube or line as needed, adhesive on base can hold tube in place.
6. Fold flap over, on top of base
7. **To Adjust:** Place finger on base and pull up on blue tab. Adjust tubes as needed, place flap back down on base.
8. **To Remove:** Swab base with water or saline as you gently peel base away from skin

**To be use by or on the order of physician
Discontinue use immediately if skin irritation occurs**