

# Product Information EZ-Hold®

- ◆ **N755 EZ-Hold®**      **White**      **100/case**
- ◆ **N757 EZ-Hold®**      **Clear**      **100/case**
- ◆ Tubing and Cannula Holder with Gentle Hydrocolloid Base and Secure Positioning Flaps

## ✓ Features and Benefits:

- ✓ **Hydrocolloid base is gentle on delicate skin**
- ✓ Leaves no residue on skin
- ✓ No skin prep required
- ✓ Clear flap allows visualization of markings
- ✓ Safe on neonatal skin
- ✓ Ideal for use with nasal cannulas, NG tubes, catheters and more
- ✓ Provides long term attachment but prevents epidermal stripping
- ✓ Easy to apply and remove
- ✓ Fold over flap reduces stress on tubes
- ✓ Latex-free
- ✓ Open and close repeatedly
- ✓ Use in NICU, PICU, Respiratory, ER, EMS, Home Health

## ➤ DFU

- Skin must be clean and dry per hospital protocol
- It is important to have skin as clean and dry as possible for proper attachment
- Moisture, oils or lotions adversely affect the adhesion of hydrocolloid
- No skin prep is required or recommended
- **DO NOT USE ALCOHOL**
- **APPLICATION:**
- Remove clear liner from base
- Place EZ-Hold® on skin where it is to be secured
- **Key: Hold in place for at least 60 seconds to ensure proper adhesion**
- Heat will increase the bond between the product and the patient
- Remove liner from flap and fold over cannula
- **REMOVAL or REPOSITION:**
- Lift flap to remove cannula
- Saturate surface of lead with water, gently peel away from skin as you swab with water
- Soap or mineral oil may also be used for removal
- **CAUTIONS:**
- Discontinue immediately if skin irritation occurs
- To be used by or on the order of a physician