

Product Information Meconium Aspirator®

- ◆ **N0101 Meconium Aspirator® Clear** **40/case**
- ◆ **N401 Meconium Aspirator® Clear** **10/box**
- ◆ **N0102 Meconium Aspirator® Yellow** **40/case**
 - ◆ Meconium Removal Device that Allows Intermittent Suctioning

✓ **Features and Benefits:**

- ✓ **Eliminates mouth suctioning of meconium**
- ✓ Attaches to an endotracheal tube
- ✓ Thumb port allows for intermittent suctioning
- ✓ Featured in the Textbook on Neonatal Resuscitation
- ✓ Used worldwide in over 20 countries
- ✓ Now available in yellow tint or original clear
- ✓ Latex-free
- ✓ Use in NICU, PICU, Respiratory, ER, EMS

➤ **DFU**

- Peel open package and remove Meconium Aspirator®
- Connect smaller end (with barbed fitting) of Meconium Aspirator® to suction line connecting tube
- Set the suction at 80mm. Hg or less
- Intubate patient with proper ET tube
- Once patient is intubated, connect larger end (15mm ID) of Meconium Aspirator® to the endotracheal tube adapter (15mm OD)
- Place thumb over suction control port to regulate suction and remove meconium
- **CAUTION:**
- **Suction for no more than two seconds at a time. It is recommended that the suctioning be done while the ET tube is being withdrawn**
- Discard after each patient use