

Product Information NeoBridge®

- ◆ **N700 NeoBridge® - (Micro - 800 < grams)** **20/case**
- ◆ **N701 NeoBridge® - (Small - 800 - 1200 grams)** **20/case**
- ◆ **N702 NeoBridge® - (Large - 1200 - 2200 grams)** **20/case**
 - ◆ **Umbilical Catheter Holder with Secure Positioning Flaps**

✓ **Features and Benefits:**

- ✓ **Attaches to patients with gentle NeoBond® hydrocolloid**
- ✓ Relieves stress on tubes and helps prevent dislodgement
- ✓ Can be used to secure two catheters
- ✓ Available in three sizes
- ✓ Flaps can be opened and closed repeatedly
- ✓ Provides long term attachment but prevents epidermal stripping
- ✓ Easy to apply and remove
- ✓ Resists breakdown from alcohol
- ✓ Clear flap allows visualization
- ✓ Leaves no residue on skin
- ✓ Latex-free
- ✓ Built in bridge saves time and standardizes application
- ✓ Use in NICU, PICU, Respiratory, ER, EMS, Home Health

➤ **DFU**

- **Skin must be clean and dry per hospital protocol**
- It is important to have skin clean and dry as possible for proper attachment
- Moisture, oils or lotions adversely affect the adhesion of NeoBond® hydrocolloid
- No skin prep is required or recommended
- **DO NOT USE ALCOHOL**
- **APPLICATION:**
- Remove backing from hydrocolloid base, slip catheter through side of base and apply base to skin
- **Key: Hold in place for at least 60 seconds to ensure proper adhesion**
- Be sure wider end is towards infant's head
- Remove backing from one flap and tack to catheter
- Repeat for remaining flap and press firmly together with loop in catheter
- If using two umbilical catheters, have one catheter come out each side of the NeoBridge® and close flaps together
- To remove one of the catheters, pull flaps apart, remove catheter and then close flaps back together
- **REMOVAL or REPOSITION:**
- Use a swab or cloth with warm water or mineral oil and gently wipe the skin as you remove the NeoBridge® from the patient's skin
- **CAUTIONS:**
- Patient must be under professional supervision
- Discontinue immediately if skin irritation occurs
- To be used by or on the order of a physician