

# Product Information NeoLead®

- ◆ **N301 NeoLead® Small** **20 sets/case**
- ◆ **N305 NeoLead® Radiolucent** **20 sets/case**
  - ◆ The Only Electrode with a HydroGel Center and Hydrocolloid Border for Long Lasting Leads

## ✓ Features and Benefits:

- ✓ **Hydrocolloid border**
- ✓ Improves sensor stability
- ✓ Prevents gel from oozing out the edges
- ✓ Safe on neonatal skin
- ✓ Provides long term attachment but prevents epidermal stripping
- ✓ Easy to apply and remove
- ✓ **HydroGel center**
- ✓ Produces excellent tracings
- ✓ Latex-free
- ✓ Won't dry out prematurely
- ✓ Use in NICU, PICU, Respiratory, ER, EMS, Home Health

## ➤ DFU

- Skin must be clean and dry per hospital protocol
- It is important to have skin as clean and dry as possible for proper attachment
- Moisture, oils or lotions adversely affect the adhesion of hydrocolloid
- No skin prep is required or recommended
- **DO NOT USE ALCOHOL**
- **APPLICATION:**
- Remove NeoLead® from liner
- Place NeoLead® on skin where it is to be secured
- **Key: Hold in place for at least 60 seconds to ensure proper adhesion**
- Heat will increase the bond between the product and the patient
- **REMOVAL:**
- Saturate surface of lead with water, gently peel away from skin as you swab with water
- Soap or mineral oil may also be used for removal
- **CAUTIONS:**
- Patient must be under professional supervision
- Discontinue immediately if skin irritation occurs
- To be used by or on the order of a physician