

Product Information Premie Prongs™

- ◆ **N82520 2.5 (7.5FR) OD 2.0cm Length** **10/Case**
- ◆ **N82525 2.5 (7.5FR) OD 2.5cm Length** **10/Case**
- ◆ **N82530 2.5 (7.5FR) OD 3.0cm Length** **10/Case**
- ◆ **N83020 3.0 (9.0FR) OD 2.0cm Length** **10/Case**
- ◆ **N83025 3.0 (10.5FR) OD 2.5cm Length** **10/Case**
- ◆ **N83030 3.0 (10.5FR) OD 3.0cm Length** **10/Case**
 - ◆ Sterile Silicone Nasopharyngeal Prongs for CPAP & NIMV without Trauma to the Nares

✓ Features and Benefits:

- ✓ **Easy to insert and remove**
- ✓ Reduce trauma to the nose and nares
- ✓ Ideal for NCPAP & IMV
- ✓ Made from soft, latex-free silicone
- ✓ New “Premie Prongs™” as short as 2cm
- ✓ Shorter prongs provide ideal placement for NCPAP or IMV
- ✓ Available in 3 different lengths and 2 different diameters
- ✓ Stays securely in place
- ✓ Use with NeoBar® for safe, simple CPAP
- ✓ Use in NICU, PICU, Respiratory, L&D, ER, EMS, Sleep Center

➤ **DFU**

- Choose Appropriate Length for Insertion
- Proper position for the tip of the airway is behind the soft palate
- A technique to determine proper insertion length is to measure from the edge of the nares to the middle of the lower eyelid and insert the airways to that depth.
- Insert airway into nares with bevel down (May lubricate per hospital protocol)
- Secure in place using a NeoBar® or per hospital protocol
- Connect 15mm adapter to oxygen source and set flow rate
- To aspirate nares, remove one side, aspirate nares and re-insert.
- Repeat for other side.
- Replace airway every 24 hours or per hospital protocol
- **CAUTION:**
- **FEDERAL (USA) LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN**
- **DO NOT CUT PRONGS**
- **DO NOT USE ALCOHOL FOR CLEANING**