

# NeoBond<sup>®</sup>

Hydrocolloid Strips Can be Used in Place of Harmful Tape



## FEATURES & BENEFITS

- Provides long term attachment but prevents epidermal stripping
- Leaves no residue on skin

<i>Cat No.</i>	<i>Size</i>	<i>Qty/Unit</i>
N741	Standard*	50 sets/box
*Each NeoBond has two 1/2" x 6" strips		



# NeoBond® Directions For Use



## Step 1

Clean and dry skin per hospital protocol.

**Note:** It is important to have skin as clean and dry as possible for proper attachment. Moisture, oils or lotions adversely affect the adhesion of hydrocolloid. No skin prep is required or recommended.

**Do not use alcohol.**



## Step 2

While clear liner is in place, warm product in hands for 60 seconds. Product may also be held under radiant warmer or heel warmer.

## Step 3

Remove NeoBond® from liner.

**SINGLE PATIENT USE ONLY**

## Step 4

Place NeoBond on skin where it is to be secured. Hold in place for at least 60 seconds to ensure proper adhesion. Heat will increase the bond between the product and the patient.

## Step 5

Removal: Saturate surface with water, gently peel away from skin as you swab with water. Soap or mineral oil may also be used for removal.

**Cautions:** Patient must be under professional supervision. Discontinue immediately if skin irritation occurs. To be used by or on the order of a physician.