

Neotech™ Chinstrap

Catalog #N785, N#786, N#787


1 Apply strap with the cut out toward baby's neck.

2 Ensure the larger strap is under the chin and the smaller strap is in front of the chin.

3 Place opening around ears.

4 Closure is designed to sit over the occiput.

5 Cut and adjust as needed. Ensure that straps are not too tight or pressing on ears.

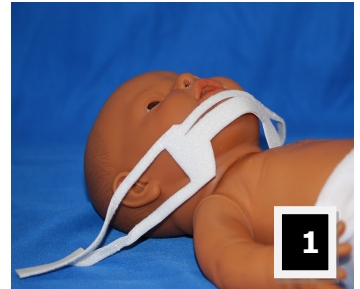

CAUTIONS:

- * Change every 3-5 days or if material becomes worn or dirty
- * Patient must be under professional supervision
- * Discontinue immediately if skin irritation occurs
- * To be used by or on the order of a physician



Neotech Products, Inc.
1-800-966-0500 • www.neotechproducts.com
27822 Fremont Court • Valencia, CA 91355