## Little/Curved Sucker Oral and Nasal Suction Device

**REF** Catalog No.

**DIRECTIONS FOR USE**  
**INSTRUCTIONS D'UTILISATION**  
**NÁVOD K POUŽITÍ**  
**GEBRAUCHSANWEISUNG**  
**ISTRUZIONI PER L'USO**  
**INSTRUCCIONES DE USO**  
**ZALECENIA DOTYCZĄCE**  
**UŻYTKOWANIA**

N204 Preemie  
N204TP Preemie  
N205 Standard  
N205TP Standard  
N208 Curved

Manufactured by  
NEOTECH PRODUCTS LLC  
28430 Witherspoon Pkwy, Valencia, CA 91355, USA  
1-800-966-0500 • neotechproducts.com

EMERGO EUROPE  
Prinsessegracht 20  
2514 AP, The Hague, The Netherlands

Made in USA

D454 Rev H

English

Little Sucker® / Curved Sucker  
Oral and Nasal Suction Device

### FEATURES & BENEFITS

- Designed for single handed suctioning
- Thumb port for intermittent suctioning
- Soft, flexible tip similar to a bulb syringe
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

### INDICATIONS FOR USE

The Little/Curved Sucker is indicated for use as a vacuum powered suction device to aspirate fluids from the mouth and nares.

### DIRECTIONS FOR USE

**Step 1** Connect barbed end of Little/Curved Sucker to suction line.

**Step 2** Set suction to appropriate pressure per hospital protocol or physician recommendation.

**Step 3** Block thumb port with thumb to begin suctioning.

**Step 4** Suction mouth or nares with intermittent suctioning.

Note: For intermittent suctioning, unblock thumb port to stop suctioning. Block thumb port again to continue suctioning.

**Step 5** Suction sterile water through Little/Curved Sucker after each use.

**Step 6** Replace Little/Curved Sucker every 24 hours or per hospital protocol, whichever is sooner.

### CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

For oral and nasal use only.

Inserting the Little/Curved Sucker too far into the mouth or nares can cause tissue trauma.

The Little/Curved Sucker should remain free of debris.

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