EZCare™ Softouch Tracheostomy Tube Holder

REF Catalog No.

N9105 5 inch
N9106 6 inch
N9107 7 inch
N9108 8 inch
N9109 9 inch
N9110 10 inch
N9111 11 inch
N926 18 inch 2-piece

**DIRECTIONS FOR USE
INSTRUCTIONS D'UTILISATION
NÁVOD K POUŽITÍ
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
INSTRUCCIONES DE USO
ZALECENIA DOTYCZAĆE
UŻYTKOWANIA**

EC REP EMERGO EUROPE
Prinsessegracht 20
2514 AP, The Hague, The Netherlands

CE

Made in USA

D491 Rev C

English
**EZCare™ Softouch
Tracheostomy Tube Holder**

FEATURES & BENEFITS

- Thick NeoFoam® Ultra for better stability
- Soft, breathable, skin friendly NeoFoam wicks perspiration
- Designed for comfort
- Lightweight and form fitting
- N926 18" can be cut to size
- Unique material helps to prevent sliding
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

INDICATIONS FOR USE

The EZCare Softouch is intended to secure a patient's tracheostomy tube. It is intended for use on pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE

Measuring

Select proper size by using EZCare Softouch measuring tape strip provided.

Place start line at flange on left side and wrap behind neck to right flange. If measurement falls between sizes, try larger size first.

Single Piece (N9105-N9111)

Step 1 With smooth, white side facing neck, insert hook tab through flange on one side of plate.

Step 2 Pull tab back over flange and firmly press white hook fabric to neckband fabric to ensure proper securement.

Step 3 Wrap EZCare around neck and insert hook tab through flange on other side of plate.

Step 4 Pull tab back over flange and firmly press white hook fabric to neckband fabric to ensure proper securement.

Note: Ensure that EZCare is taught, but not overly stretched. Adjust as needed.

Step 5 Replace EZCare Softouch every 5 days or per hospital protocol, whichever is sooner.

Replace immediately if hook tab is worn or visibly soiled.

2-piece (N926)

Step 1 Separate two pieces.

Step 2 With smooth, white side facing neck, insert hook tabs through flanges on each side of plate.

Step 3 Pull tab back over flange and firmly press white hook fabric to neckband fabric to ensure proper securement.

Step 4 Wrap both sides of EZCare around neck and bring together.

Step 5 Determine proper length, pull longer piece away from neck and cut excess.

Note: Ensure that skin will not be exposed to hook fabric secure tab.

Step 6 Bring both sides back together and firmly press white hook fabric secure tab to neckband fabric to ensure proper securement.

Note: Ensure that EZCare is taught, but not overly stretched. Adjust as needed.

Step 7 Replace EZCare Softouch every 5 days or per hospital protocol, whichever is sooner.

Replace immediately if hook tab is worn or visibly soiled.

CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

Replace immediately if hook tab is worn or visibly soiled. Worn tab may cause decannulation.

Check regularly to prevent disengagement.

Discontinue immediately if skin irritation occurs.

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