

# Suction Caddy™

Suction Device Holder



## FEATURES AND BENEFITS

- Easy to use and attaches to most smooth surfaces
- Keeps suction devices and tubing close at hand and off the floor
- Convenient, designated write-on area for patient name and date
- Clear plastic allows for better visibility
- Tubing anchor helps to reduce noise
- Large suction cups provide strong hold and allow for easy removal
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

Cat No.	Item	Qty/Unit
N245	Suction Caddy	20/box



Made in USA

U.S. Patent #7,601,001

# Suction Caddy™

## Suction Device Holder

### INTENDED USE

---

The Suction Caddy is intended to hold and store a suction device.

### DIRECTIONS FOR USE

---

#### Step 1

Affix Suction Caddy to a smooth, flat surface by pressing nubs of suction cups firmly.

Note: Ensure that compartment opening is facing up.

#### Step 2

2A) If disconnecting suction device from tubing, place device in Suction Caddy compartment and slip open end of tubing over tubing anchor.

or

2B) If keeping suction device and tubing connected, place connected device into Suction Caddy compartment.

#### Step 3

To remove Suction Caddy from surface, lift suction cup tabs.

#### Step 4

Replace Suction Caddy every 72 hours or per hospital protocol, whichever is sooner.

Caution: Replace immediately if visibly soiled.

### GENERAL INFORMATION

---

#### Product Materials

Suction Caddy: Plastic K-Resin® Copolymer

Suction cups: Vinyl

Made with natural rubber latex: No

Made with plasticizer DEHP: No

Consult MSDS for complete material information.

#### Size

Width: 50 mm

Height: 110 mm (130 mm including tubing anchor)

Compartment diameter: 20 mm

#### Packaging

Individually packaged: Yes

Package type: Polyethylene bag

Dispenser box: Yes

*Visit [neotechproducts.com](http://neotechproducts.com) for the most up-to-date information or to download the Material Safety Data Sheet (MSDS).*

*Polyethylene bags are biodegradable. Box is printed with environmentally friendly soy ink on recyclable material.*

### CAUTIONS

---

The Suction Caddy is designed for use with Neotech suction devices specifically. It may not accommodate other suction devices.

The Suction Caddy and suction cups should be kept clean and free of debris and fluids.

Replace immediately if visibly soiled or damaged.

Single patient use only.



For more information or to order your FREE sample call  
**1-800-966-0500** or visit **[neotechproducts.com](http://neotechproducts.com)**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355