


NeoSpoon[™]

Disposable Measuring Spoon

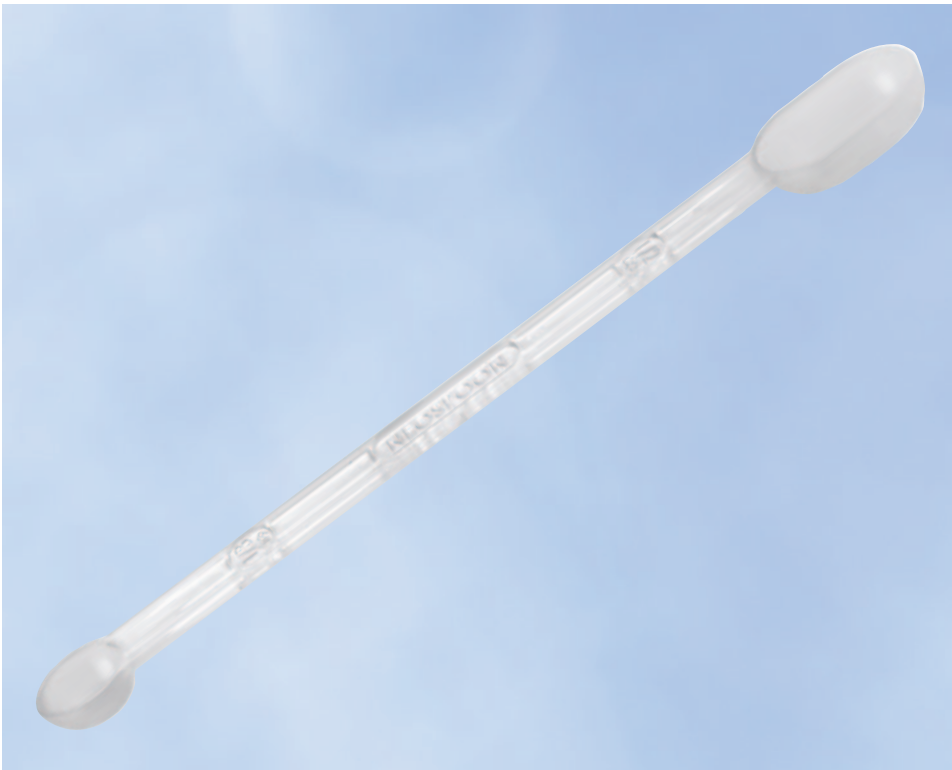

FEATURES AND BENEFITS

- Dual spoon design (1/8 tsp and 1/2 tsp)
- Engineered for quick, accurate measuring
- Dispense exact volume in multiples of 1/8 tsp
- Ideal for feeding preparation
- Environmentally friendly, biodegradable material
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

Cat No.	Volume	Qty/Unit
N761	1/8 & 1/2 tsp	100/box



Made in USA

NeoSpoon™

Disposable Measuring Spoon

INDICATIONS FOR USE

The NeoSpoon is intended for use as a measuring device

DIRECTIONS FOR USE

Step 1

Scoop and level substance.

Note: Leveling is essential for accurate measurement.

Step 2

Measure desired volume of single substance.

Note: Do not use single NeoSpoon for multiple substances.

Step 3

Dispose after each use.

Note: If used to measure hazardous substance, dispose in proper hazardous waste container.

GENERAL INFORMATION

Product Material

NatureWorks® Polylactide Resin
Made with natural rubber latex: No
Made with plasticizer DEHP: No
Consult MSDS for complete material information.

Size

Length: 168 mm
Volume: 1/8 tsp and 1/2 tsp

Packaging

Individually packaged: Yes
Package type: Polyethylene bag
Dispenser box: Yes

Visit neotechproducts.com for the most up-to-date information or to download the Material Safety Data Sheet (MSDS).

Polyethylene bags are biodegradable. Boxes are printed with environmentally friendly soy ink on recyclable material.

CAUTIONS & WARNINGS

If used to measure hazardous substance, dispose in proper hazardous waste container.

Single substance use only.



For more information or to order your FREE sample call
1-800-966-0500 or visit **neotechproducts.com**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355