


NeoFoam® Ultra

EZCare™ Softouch

Tracheostomy Tube Holder

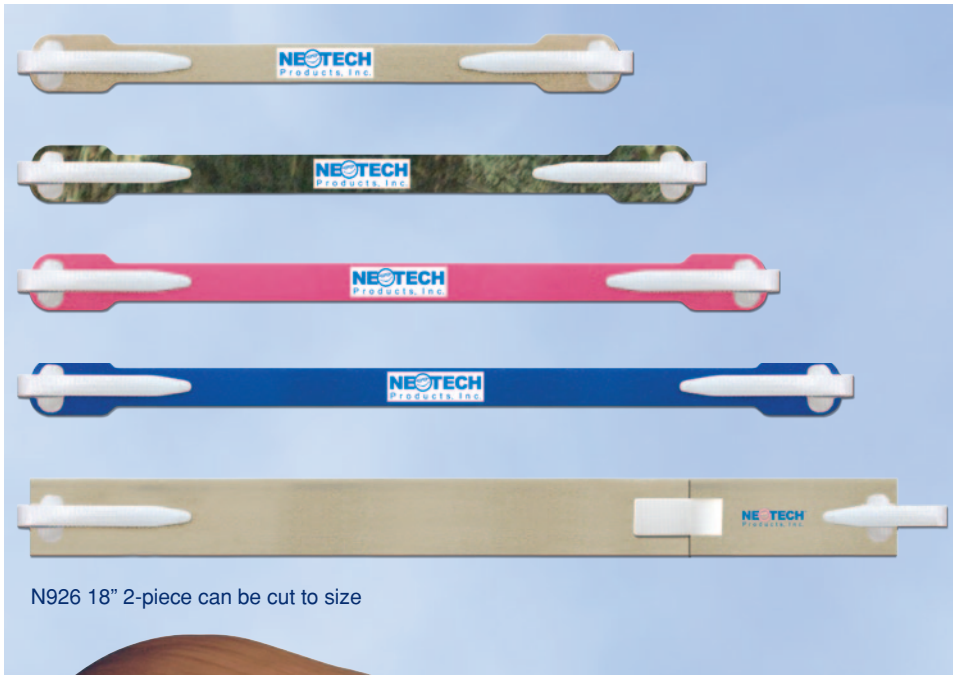

N926 18" 2-piece can be cut to size

FEATURES AND BENEFITS

- Thick NeoFoam® Ultra for better stability
- Soft, breathable, skin friendly NeoFoam wicks perspiration
- Designed for comfort
- Lightweight and form fitting
- N926 18" can be cut to size
- Unique material helps to prevent sliding
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

Cat No.	Item	Qty/Unit
N9105	5" Single piece	20/box
N9106	6" Single piece	20/box
N9107	7" Single piece	20/box
N9108	8" Single piece	20/box
N9109	9" Single piece	20/box
N9110	10" Single piece	20/box
N9111	11" Single piece	20/box
N926	18" 2-piece (tan only)	20/box

Tan=TN, Camouflage=CO, Pink=PK, Blue=BL



Made in USA

EZCare™ Softouch

Tracheostomy Tube Holder

DIRECTIONS FOR USE

Measuring

Select proper size by using EZCare Softouch measuring tape strip provided.

Place start line at flange on left side and wrap behind neck to right flange. If measurement falls between sizes, try larger size first.

Single Piece (N9105-N9111)

Step 1

With smooth, white side facing neck, insert hook tab through flange on one side of plate.

Step 2

Pull tab back over flange and firmly press white hook fabric to neckband fabric to ensure proper securement.

Step 3

Wrap EZCare around neck and insert hook tab through flange on other side of plate.

Step 4

Pull tab back over flange and firmly press white hook fabric to neckband fabric to ensure proper securement.

Note: Ensure that EZCare is taught, but not overly stretched. Adjust as needed.

Step 5

Replace EZCare Softouch every 5 days or per hospital protocol, whichever is sooner.

Replace immediately if hook tab is worn or visibly soiled.

2-piece (N926)

Step 1

Separate two pieces.

Step 2

With smooth, white side facing neck, insert hook tabs through flanges on each side of plate.

Step 3

Pull tab back over flange and firmly press white hook fabric to neckband fabric to ensure proper securement.

Step 4

Wrap both sides of EZCare around neck and bring together.

Step 5

Determine proper length, pull longer piece away from neck and cut excess.

Note: Ensure that skin will not be exposed to hook fabric secure tab.

Step 6

Bring both sides back together and firmly press white hook fabric secure tab to neckband fabric to ensure proper securement.

Note: Ensure that EZCare is taught, but not overly stretched. Adjust as needed.

Step 7

Replace EZCare Softouch every 5 days or per hospital protocol, whichever is sooner.

Replace immediately if hook tab is worn or visibly soiled.

INDICATIONS FOR USE

The EZCare Softouch is intended to secure a patient's tracheostomy tube. It is intended for use on pediatric (neonates, infants and children) patients.

GENERAL INFORMATION

Product Materials

NeoFoam Ultra, hook fabric

Made with natural rubber latex: No

Made with plasticizer DEHP: No

Consult MSDS for complete material information.

Size

N9105 5" Single piece

N9106 6" Single piece

N9107 7" Single piece

N9108 8" Single piece

N9109 9" Single piece

N9110 10" Single piece

N9111 11" Single piece

N926 18" 2-piece (tan only)

Packaging

Individually packaged: Yes

Package type: Polyethylene bag

Dispenser box: Yes

Visit neotechproducts.com for the most up-to-date information or to download the Material Safety Data Sheet (MSDS).

Polyethylene bags are biodegradable. Boxes are printed with environmentally friendly soy ink on recyclable material.

CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

Replace immediately if hook tab is worn or visibly soiled. Worn tab may cause decannulation.

Check regularly to prevent disengagement.

Discontinue immediately if skin irritation occurs.

Single patient use only.



**For more information or to order your FREE sample call
1-800-966-0500 or visit neotechproducts.com**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355