


**NeoLead**<sup>®</sup>  
 ECG Electrodes

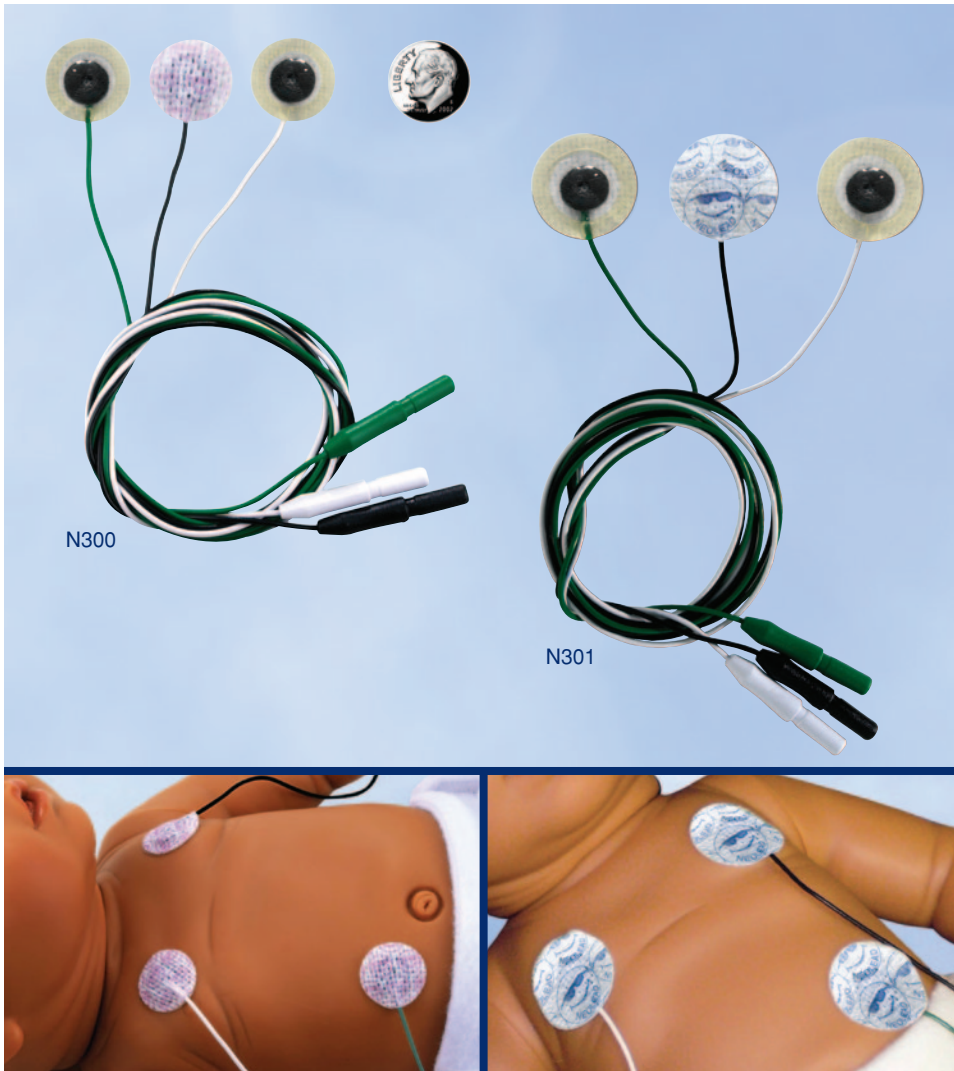

**FEATURES AND BENEFITS**

- Pre-wired electrodes
- Hydrogel center
- NeoBond<sup>®</sup> Hydrocolloid border designed to:
  - prevent hydrogel from oozing or drying prematurely
  - provide sensor stability for better tracings
  - minimize slipping
- Multi-colored wires for easier identification
- Not made with natural rubber latex or plasticizer DEHP
- Sets of three individually packaged

Cat No.	Item	Size	Qty/Unit
N300	Mini	3/4" circle	20 sets/box
N301	Standard	1" Circle	20 sets/box



Made in USA

U.S. Patent #6,795,722

# NeoLead®

## ECG Electrodes

### INDICATIONS FOR USE

The NeoLead® ECG Electrodes are intended for ECG monitoring. They are intended for use on pediatric (neonates, infants and children) patients.

### DIRECTIONS FOR USE

#### Preparation

Clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. Electrodes will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

#### Application

##### Step 1

Before removing liner, warm the lead in hands for 60 seconds. May also be held under radiant warmer or heel warmer for 10 to 15 seconds.

##### Step 2

Peel and discard liner.

##### Step 3

Place the lead on skin and hold in place for 60 seconds to ensure proper adhesion.

#### Removal

##### Step 1

Replace NeoLead every 5 days or per hospital protocol, whichever is sooner.

##### Step 2

Saturate the lead with water or saline.

##### Step 3

Slowly peel the lead away from the skin as you swab with water or saline.

### GENERAL INFORMATION

#### Product Materials

Electrode: Silver silver chloride, hydrogel, NeoBond Hydrocolloid, non-woven fabric  
Wire: Copper, polypropylene  
Made with natural rubber latex: No  
Made with plasticizer DEHP: No

#### Size

N300 diameter: 19 mm  
N301 diameter: 26 mm  
Wire length: 61 cm

#### Packaging

Sets individually packaged: Yes  
Package type: Pouch  
Dispenser box: Yes

*Visit [neotechproducts.com](http://neotechproducts.com) for the most up-to-date information.*

*Pouches are made from paper, polyethylene and foil. Boxes are printed with environmentally friendly soy ink on recyclable material.*

### CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

NeoLead® ECG Electrodes should only be used while patient is under continuous direct supervision of healthcare professionals.

Electrodes will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Do not use NeoLead (N300, N301) during x-ray or MR procedures.

Discontinue immediately if skin irritation occurs.

Disposable. Do not reuse.

Single patient use only.



**For more information or to order your FREE sample call  
1-800-966-0500 or visit [neotechproducts.com](http://neotechproducts.com)**

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355