


*New*  
**NeoPockets<sup>™</sup>**  
 Personal Incubator Organizer

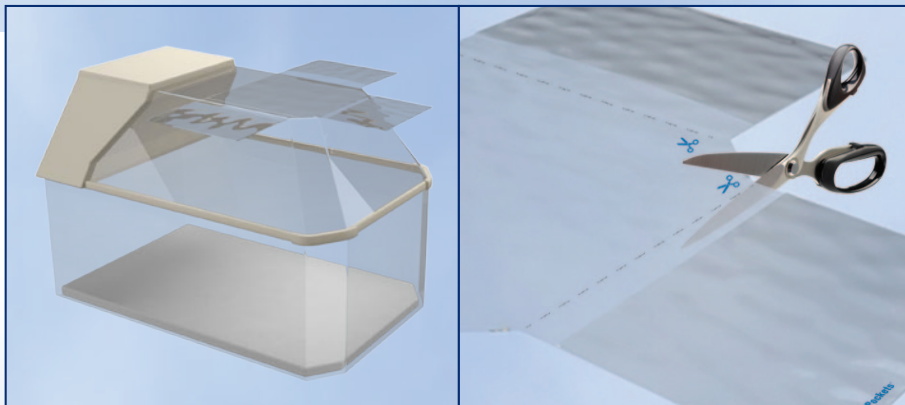

**FEATURES AND BENEFITS**

- Apply and remove repeatedly with no adhesive residue
- Crystal clear vinyl allows for visibility of patient
- Adheres to most smooth surfaces
- Three pockets for:
  - Patient info
  - Thermometer
  - Family photos and more
- Perforated guidelines to cut and customize
- Contours to a variety of incubator models
- Not made with natural rubber latex or plasticizer DEHP

Cat No.	Item	Qty/Unit
N7325	NeoPockets	20/box



Patent Pending

# NeoPockets™

## Personal Incubator Organizer

### INTENDED USE

---

Intended to store miscellaneous information and accessories.

### DIRECTIONS FOR USE

---

#### Step 1

Unfold NeoPockets.

#### Step 2

Peel away from paper backing; discard backing.

#### Step 3

Lay middle section on top of incubator and align edges.

#### Step 4

Fold sides and front flap down over incubator.

Note: Pockets can be cut off to meet your needs. Be sure to distribute weight evenly.

#### Step 5

Remove or reposition as needed.

### GENERAL INFORMATION

---

#### Product Materials

Static cling vinyl

Made with natural rubber latex: No

Made with plasticizer DEHP: No

#### Size

25 in. x 16 in. (63.5 cm x 40.5 cm)

Pocket 1: 6.5 in. x 5 in. (165 mm x 127 mm)

Pocket 2: 2.3 in. x 5 in. (58 mm x 127 mm)

Pocket 3: 7.8 in. x 5.5 in. (198 mm x 140 mm)

Pocket 4 8.25 in. x 6.5 in. (209.5 x 165 mm)

### CAUTIONS & WARNINGS

---

Discard if static cling becomes ineffective.

Distribute weight evenly.



For more information or to order your FREE sample call  
**1-800-966-0500** or visit [neotechproducts.com](http://neotechproducts.com)

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355