NeoShades® with Headstrap Phototherapy Eye Shields

REF Catalog No.

N727 Micro
N728 Small
N729 Large

DIRECTIONS FOR USE
INSTRUCTIONS D'UTILISATION
NÁVOD K POUŽITÍ
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
INSTRUCCIONES DE USO
ZALECENIA DOTYCZĄCE
UŻYTKOWANIA

Manufactured by
NEOTECH PRODUCTS LLC
28430 Witherspoon Pkwy, Valencia, CA 91355, USA
1-800-966-0500 • neotechproducts.com

EMERGO EUROPE
Prinsessegracht 20
2514 AP, The Hague, The Netherlands

Made in USA

D458 Rev F

PPE Regulation (EU) 2016/425

Notified Body: BSI Group (0086),
Kitemark Court, Davy Avenue, Knowlhill, Milton
Keynes, MK5 8PP, UK.

English

NeoShades® with Headstrap
Phototherapy Eye Shields

FEATURES & BENEFITS

- Soft, padded eye shield material with cute sunglasses design
- Ocular pockets designed for patient comfort and safety
- Soft, patented NeoFoam® headstrap holds NeoShades in place
- Adjustable headstrap engineered to minimize slipping and help reduce head molding
- Unique headstrap shape designed to ensure ideal fit
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

INDICATIONS FOR USE

The NeoShades are indicated for use as eye protection during phototherapy sessions. They are intended for use on pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE

- Step 1** Select proper size.
- Step 2** Place headstrap behind head with smooth side of foam against head.
- Step 3** Cover eyes with NeoShades.
- Step 4** Gently attach hook fabric on end of headstrap to NeoShades.
- Repeat for opposite side.
- Note: Straps can be trimmed for proper fit.

CAUTIONS & WARNINGS

The NeoShades are not intended for use on adults.

Federal law restricts this device to sale by or on the order of a physician.

NeoShades should only be used while the patient is under the continuous, direct supervision of healthcare professionals.

Single patient use only.

General transportation and storage practices are applicable

Declaration of conformity can be viewed on the Neotech website:
www.neotechproducts.com

Product expiration date is printed on the immediate packaging as well as the dispenser box of the product

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