


NeoHeart[™] ECG Pad

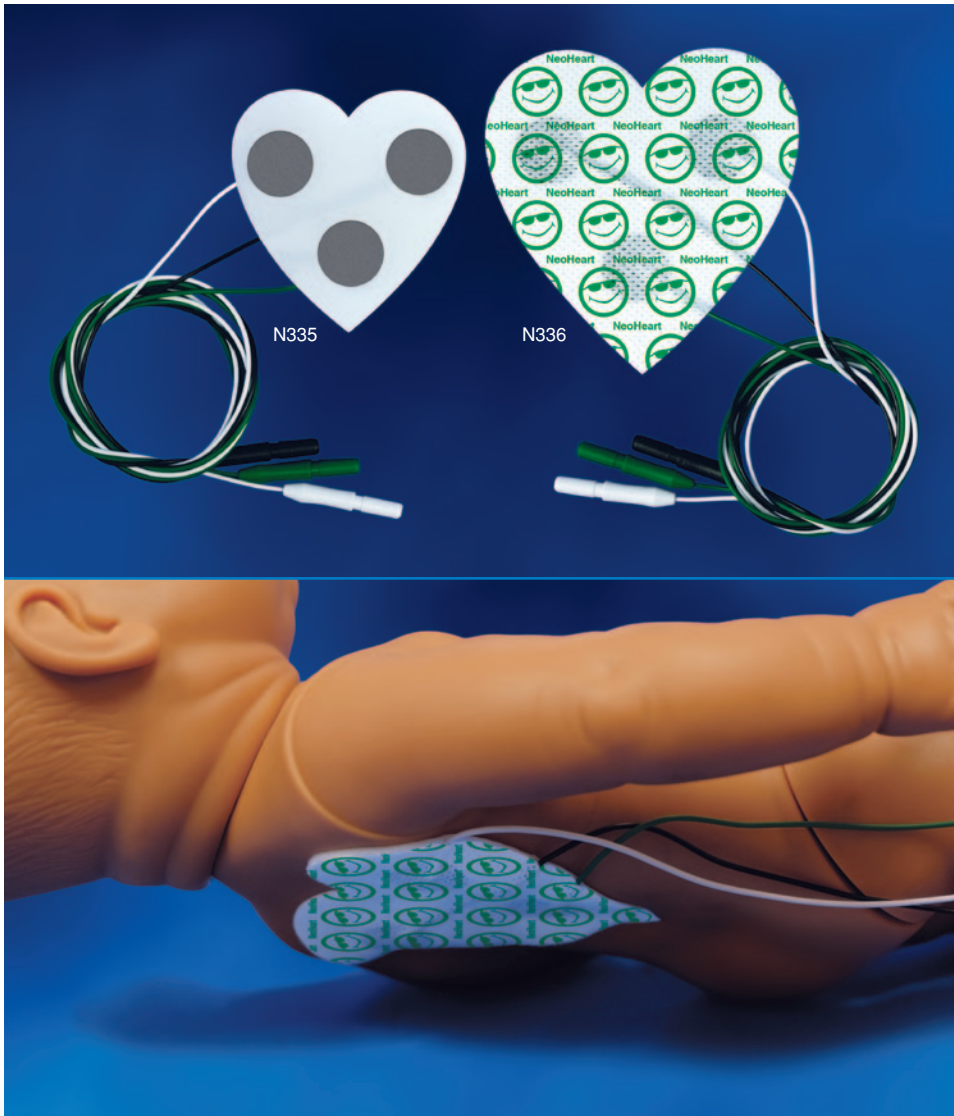

FEATURES AND BENEFITS

- 3-in-1 pre-wired electrode array
- Pre-measured sensor position
- Radiolucent material nearly invisible on x-rays
- One touch application ideal for use in delivery room
- Larger surface area for enhanced adherence
- Engineered for back application, allowing chest to remain exposed
- Multi-colored wires for easier identification
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

Cat No.	Size	Qty/Unit
N335	Small	20 sets/box
N336	Large	20 sets/box



Made in USA

Patents Pending

NeoHeart™

ECG Pad

INDICATIONS FOR USE

NeoHeart is a set of three attached ECG electrodes used on neonates, infants, and pediatric patients.

DIRECTIONS FOR USE

Application

Step 1

Peel and discard liner. Apply immediately after liner is removed.

Step 2

Place NeoHeart on patient's back and connect to ECG monitor cable. The pad should be oriented so the point of the heart is pointing toward lower extremities.

Step 3

Confirm quality of ECG tracing. If not achieving quality tracing or skin adherence, remove excess moisture from skin and reapply.

CAUTION: If placed on patient's chest, the white wire electrode will be on patient's left side. Insert into monitor accordingly.

Removal

Step 1

Remove NeoHeart within 24 hours.

Step 2

Saturate NeoHeart with water or saline.

Step 3

Slowly peel NeoHeart away from skin as you swab skin with water or saline.

Replace with standard electrodes as directed by physician.

GENERAL INFORMATION

Product Materials

Heart: Non-woven fabric, medical grade acrylic
Electrode: Carbon, silver silver chloride, hydrogel
Wire: Carbon, polypropylene
Made with natural rubber latex: No
Made with plasticizer DEHP: No

Size

N335: 61 x 63.5 mm

N336: 82 x 84 mm

Wire length: 61 cm

Packaging

Individually packaged: Yes

Package type: Pouch

Dispenser box: Yes

Visit neotechproducts.com for the most up-to-date information.

Pouches are made from paper, polyethylene and foil.

CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

NeoHeart ECG Pad should only be used while patient is under continuous direct supervision of healthcare professionals.

NeoHeart is designed for back application. If placed on patient's chest, the white wire electrode will be on patient's left side. Insert into monitor accordingly.

Do not apply to open wounds or broken skin.

Discontinue immediately if skin irritation occurs.

Discard NeoHeart 24 hours after opening pouch.

Disposable. Do not reuse.

Single patient use only.



**For more information or to order your FREE sample call
1-800-966-0500 or visit neotechproducts.com**

Neotech Products LLC • 28430 Witherspoon Pkwy • Valencia, CA 91355, USA