NEOTECH[®] Products LLC

Little Sucker[®] NeoSucker[®]

REF Catalog No.

| | |
|--------|--------|
| N204 | N207 |
| N204C | N208 |
| N204TP | N215 |
| N205 | N224 |
| N205C | N224TP |
| N205TP | N225 |
| N206 | N225TP |

DIRECTIONS FOR USE
INSTRUCTIONS D'UTILISATION
ISTRUZIONI PER L'USO
INSTRUCCIONES DE USO
INSTRUÇÕES DE UTILIZAÇÃO
GEBRAUCHSANWEISUNG
NÁVOD K POUŽITÍ
ZALECENIA DOTYCZĄCE
UŻYTKOWANIA
إرشادات الاستخدام

 Manufactured by
NEOTECH PRODUCTS LLC
28430 Witherspoon Pkwy, Valencia, CA 91355, USA
1-800-966-0500 • neotechproducts.com

EC REP EMERGO EUROPE
Prinsessegracht 20
2514 AP, The Hague, The Netherlands

CE

Made in USA

D454 Rev J

English

INDICATIONS FOR USE

The Little Sucker/NeoSucker is indicated for use as a tip for suction tubing to aspirate fluids from the mouth and nares.

The Little Sucker Nasal Tip is indicated for use as a tip for suction tubing to aspirate fluids from the nares.

The Little Sucker Cover is indicated for use to protect the Little Sucker Oral and Nasal Suction Device.

DIRECTIONS FOR USE

Step 1 If applicable, unplug thumb port and slide suction tip out of cover.

Step 2 Connect barbed end of suction tip to suction line.

Step 3 Set suction to appropriate pressure per hospital protocol or physician recommendation.

Step 4 Block thumb port with thumb to begin suctioning.

Step 5 Suction mouth or nares with intermittent suctioning.

Note: For intermittent suctioning, unblock thumb port to stop suctioning. Block thumb port again to continue suctioning.

Step 6 Suction sterile water through suction tip after each use.

Step 7 If applicable, reinsert suction tip into cover.

Step 8 If applicable, insert port plug of cover into thumb port of suction tip.

Step 9 Replace suction tip/cover every 24 hours or per hospital protocol, whichever is sooner.

CAUTIONS & WARNINGS

For oral or nasal use only.

Inserting the suction tip too far into the mouth or nares can cause tissue trauma.

The suction tip/cover should remain free of debris.

The Little Sucker Cover is designed for use with the Little Sucker Oral and Nasal Suction Device specifically. It may not accommodate other suction devices. It is not intended for use with the Little Sucker 2-piece or Little Sucker Nasal Tip.

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