NeoSmile™

Temperature Probe Cover

REF Catalog No.

N731

DIRECTIONS FOR USE

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UŻYTKOWANIA

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 Manufactured by
NEOTECH PRODUCTS LLC
28430 Witherspoon Parkway, Valencia, CA 91355, USA
1-800-966-0500 • neotechproducts.com

EC **REP** EMERGO EUROPE
Prinsessegracht 20
2514 AP, The Hague, The Netherlands

Made in USA

D480 Rev E

English

NeoSmile™

Temperature Probe Cover

INDICATIONS FOR USE

The NeoSmile is intended to cover and secure a temperature probe.

DIRECTIONS FOR USE

Preparation

Clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. NeoSmile will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

Application

Step 1 Before removing liner, warm NeoSmile in hands for 60 seconds. May also be held under radiant or heel warmer for 10 to 15 seconds

Step 2 Peel and discard liner. Place NeoSmile over probe and skin and hold in place for 60 seconds to ensure proper adhesion.

Removal

Step 1

Replace NeoSmile every 5 days or per hospital protocol, whichever is sooner.

Step 2

Saturate NeoSmile with water or saline.

Step 3

Slowly peel NeoSmile away from skin as you swab with water or saline.

CAUTIONS & WARNINGS

Discontinue immediately if skin irritation occurs.

NeoSmile will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

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