## NeoBond® Hydrocolloid Adhesive Strips

<b>REF</b> Catalog No.	<b>DIRECTIONS FOR USE</b>
N741 Pre-cut Strips N742 Roll	<b>INSTRUCTIONS D'UTILISATION</b> <b>ISTRUZIONI PER L'USO</b> <b>INSTRUCCIONES DE USO</b> <b>INSTRUÇÕES DE UTILIZAÇÃO</b> <b>GEBRAUCHSANWEISUNG</b> <b>NÁVOD K POUŽITÍ</b> <b>ZALECENIA DOTYCZĄCE</b> <b>UŻYTKOWANIA</b> <b>إرشادات الاستخدام</b>

 Manufactured by  
NEOTECH PRODUCTS LLC  
28430 Witherspoon Parkway, Valencia, CA 91355, USA  
1-800-966-0500 • neotechproducts.com

**EC REP** EMERGO EUROPE  
Prinsessegracht 20  
2514 AP, The Hague, The Netherlands

Made in USA

D490 Rev E

English

NeoBond®  
Hydrocolloid Adhesive Strips

### INDICATIONS FOR USE

NeoBond Hydrocolloid Adhesive Strips are intended for general use as a medical adhesive.

### DIRECTIONS FOR USE

#### **Preparation**

Clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. NeoBond will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

#### **Application**

**Step 1** Before removing liner, warm NeoBond in hands for 60 seconds. May also be held under radiant or heel warmer for 10 to 15 seconds.

**Step 2** Peel and discard liner. Wrap NeoBond Strip around skin and product and hold in place for 60 seconds to ensure proper adhesion.

#### **Removal**

**Step 1** Replace NeoBond Strip every 5 days or per hospital protocol, whichever is sooner.

**Step 2** Saturate NeoBond Strip with water or saline.

**Step 3** Slowly peel NeoBond Strip away from skin as you swab with water or saline.

### CAUTIONS & WARNINGS

Discontinue immediately if skin irritation occurs.

NeoBond will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

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