# NEOTECH<sup>®</sup> Products LLC

## Hold-A-Line™ Catheter and Tubing Holder

<span><span></span></span> <b>[REF]</b> Catalog No. N705	<b>DIRECTIONS FOR USE</b> <b>INSTRUCTIONS D'UTILISATION</b> <b>ISTRUZIONI PER L'USO</b> <b>INSTRUCCIONES DE USO</b> <b>INSTRUÇÕES DE UTILIZAÇÃO</b> <b>GEBRAUCHSANWEISUNG</b> <b>NÁVOD K POUŽITÍ</b> <b>ZALECENIA DOTYCZĄCE</b> <b>UŻYTKOWANIA</b> <b>إرشادات الاستخدام</b>
	

<span><span></span></span> <b>[EC]</b> <b>[REP]</b>	<b>EMERGO EUROPE</b> Prinsessegracht 20 2514 AP, The Hague, The Netherlands
---	---

D494 Rev E

*English*  
**Hold-A-Line™**  
**Catheter and Tubing Holder**

### **INDICATIONS FOR USE**

The Hold-A-Line is intended to anchor and support umbilical catheters and tubing. It is intended for use on pediatric (neonates, infants and children) patients.

### **DIRECTIONS FOR USE**

#### **Preparation**

After catheter or tube is in place, clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. Hold-A-Line will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

#### **Application**

**Step 1** Before removing liners, warm Hold-A-Line in hands for 60 seconds. May also be held under radiant or heel warmer for 10 to 15 seconds.

**Step 2** Discard white, paper liner, then peel and discard clear, plastic liner from hydrocolloid base.

**Step 3** Apply base to skin and hold in place for 60 seconds to ensure proper adhesion.

**Step 4** Peel and discard frosted liner and adhere catheter or tube to base.

**Step 5** Fold flap over catheter or tube, align with base and press firmly together, enclosing catheter or tube between adhesives.

**Step 6** Lift flap as needed to reposition or remove catheter or tube.

#### **Removal**

**Step 1** Replace Hold-A-Line every 5 days or per hospital protocol, whichever is sooner.

**Step 2** Lift flap to remove catheter or tube.

**Step 3** Saturate Hold-A-Line with water or saline.

**Step 4** Slowly peel Hold-A-Line away from skin as you swab with water or saline.

### **CAUTIONS & WARNINGS**

Federal law restricts this device to sale by or on the order of a physician.

Hold-A-Line should only be used while the patient is under the continuous, direct supervision of trained healthcare professionals.

Discontinue immediately if skin irritation occurs.

Hold-A-Line will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

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