


**NeoHug<sup>®</sup>**  
 Utility Device Holder

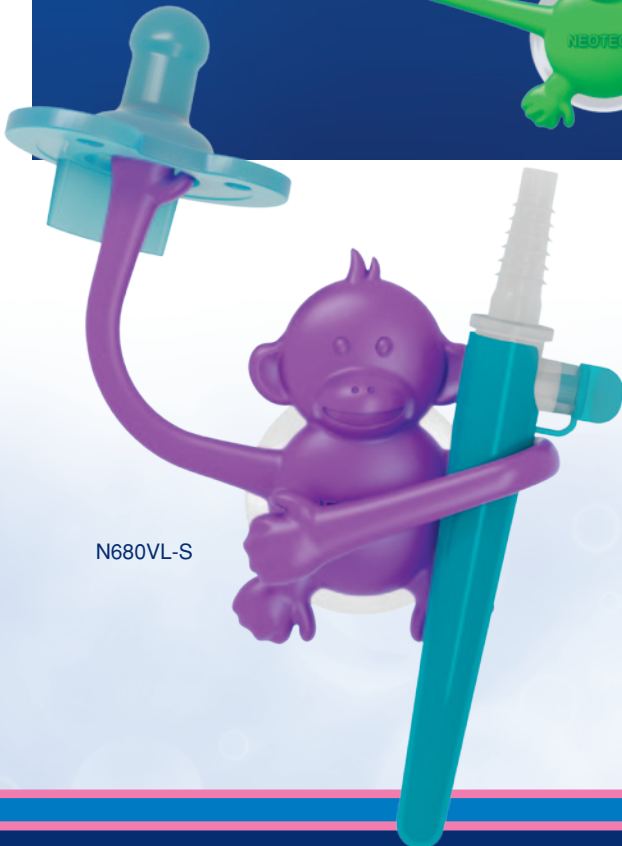

**FEATURES AND BENEFITS**

- Easy to use and attaches to most smooth surfaces
- Large suction cup for strong hold and easy removal
- Holds under heat and humidity
- Reposition repeatedly with no adhesive residue
- Bendable arms to hold a variety of light weight items
- Ideal for holding pacifier, suction tip, tape and more
- Playful chimp design available in five bright colors
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

Cat No.	Color	Qty/Unit
N680BL-S	Blue	25/box
N680BO-S	Bright Orange	25/box
N680BP-S	Bright Pink	25/box
N680GR-S	Green	25/box
N680VL-S	Violet	25/box
N680A-S	Assorted (5 of each color)	25/box



**Clinician Invented**

Patent Pending

# NeoHug®

## Utility Device Holder

### INDICATIONS FOR USE

---

NeoHug is intended to support, organize or hold non-operating medical devices.

### DIRECTIONS FOR USE

---

#### Step 1

Remove NeoHug from package and discard biodegradable bag.

#### Step 2

Affix NeoHug by pressing center of suction cup firmly against smooth, flat surface.

Note: If mounting in high humidity (i.e. inside incubators), wipe the surface of excess moisture before mounting.

#### Step 3

Bend arms as needed to hold desired items.

#### Step 4

Replace NeoHug every 14 days or per hospital protocol, whichever is sooner.

To remove NeoHug from surface, lift suction cup tab.

Replace immediately if visibly soiled or damaged.

### GENERAL INFORMATION

#### Product Materials

NeoHug (chimp): TPE, ABS, stainless steel

Suction cup: PVC

Made with natural rubber latex: No

Made with plasticizer DEHP: No

#### Size

Bendable arm span: 19.5 cm

Suction cup diameter: 45 mm

#### Weight Limit

Holds up to 113 gr (4 oz) with both arms.

#### Packaging

Individually packaged: Yes

Package type: Biodegradable polyethylene bag

*Visit [neotechproducts.com](http://neotechproducts.com) for the most up-to-date information.*

### CAUTIONS

---

The suction cup should be kept clean and free of debris and fluids.

Replace if suction cup is no longer adhering effectively.

Replace immediately if visibly soiled or damaged.

Single patient use only.

**For more information or to order your FREE sample call  
1-800-966-0500 or visit [neotechproducts.com](http://neotechproducts.com)**

Neotech Products LLC • 28430 Witherspoon Parkway • Valencia, CA 91355