| | |
|---|-------------------|
|  | |
| Part # | D402 |
| Revision | N |
| Description | SUCTION CADDY DFU |
| Artwork # | AW4402 |
| Material # (PM, DM, LM) & description | DM466 |
| Effective date | |
| CO # | |

NEOTECH
Products LLC

Suction Caddy™
Suction Device Holder

REF Catalog No. N245

Manufactured by
NEOTECH PRODUCTS LLC
28430 Witherspoon Parkway , Valencia, CA 91355, USA
1-800-966-0500 • neotechproducts.com

EMERGO EUROPE
Westervoortsedijk 60
6827 AT, Arnhem, The Netherlands

MD Medical Device

CE

D402 Rev N 2025.02.10

English
Suction Caddy™ Suction Device Holder
INDICATIONS FOR USE
The Suction Caddy is intended to hold and store a suction device.
DIRECTIONS FOR USE
Step 1 Affix Suction Caddy to a smooth, flat surface by pressing nubs of suction cups firmly.
Note: Ensure that compartment opening is facing up.
Step 2 2A) If disconnecting suction device from tubing, place device in Suction Caddy compartment and slip open end of tubing over tubing anchor. or 2B) If keeping suction device and tubing connected, place connected device into Suction Caddy compartment.
Step 3 To remove Suction Caddy from surface, lift suction cup tabs.
Step 4 Replace Suction Caddy every 72 hours or per hospital protocol, whichever is sooner.
Caution: Replace immediately if visibly soiled.
CAUTIONS
The Suction Caddy is designed for use with Neotech suction devices specifically. It may not accommodate other suction devices.
The Suction Caddy and suction cups should be kept clean and free of debris and fluids.
Replace immediately if visibly soiled or damaged.
Single patient use only.
European Union: MDR EU 2017/745
Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.

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