# **Suction Caddy™** Suction Device Holder

REF Catalog No. N245

**DIRECTIONS FOR USE**  
**INSTRUCTIONS D'UTILISATION**  
**ISTRUZIONI PER L'USO**  
**INSTRUCCIONES DE USO**  
**INSTRUÇÕES DE UTILIZAÇÃO**  
**GEBRAUCHSANWEISUNG**  
**NÁVOD K POUŽITÍ**  
**ZALECENIA DOTYCZĄCE**  
**UŻYTKOWANIA**  
**ANVÄNDARINSTRUKTIONER**  
**KÄYTTÖOHJEET**  
**إرشادات الاستخدام**

 Manufactured by  
NEOTECH PRODUCTS LLC  
28430 Witherspoon Parkway, Valencia, CA 91355, USA  
1-800-966-0500 • neotechproducts.com

EC  REP EMERGO EUROPE  
Prinsessegracht 20  
2514 AP, The Hague, The Netherlands

MD Medical Device

Made in USA

D402 Rev K

*English*  
**Suction Caddy™**  
Suction Device Holder

## INDICATIONS FOR USE

The Suction Caddy is intended to hold and store a suction device.

## DIRECTIONS FOR USE

**Step 1** Affix Suction Caddy to a smooth, flat surface by pressing nubs of suction cups firmly.

Note: Ensure that compartment opening is facing up.

### **Step 2**

**2A)** If disconnecting suction device from tubing, place device in Suction Caddy compartment and slip open end of tubing over tubing anchor.

or

**2B)** If keeping suction device and tubing connected, place connected device into Suction Caddy compartment.

**Step 3** To remove Suction Caddy from surface, lift suction cup tabs.

**Step 4** Replace Suction Caddy every 72 hours or per hospital protocol, whichever is sooner.

Caution: Replace immediately if visibly soiled.

## CAUTIONS

The Suction Caddy is designed for use with Neotech suction devices specifically. It may not accommodate other suction devices.

The Suction Caddy and suction cups should be kept clean and free of debris and fluids.

Replace immediately if visibly soiled or damaged.

Single patient use only.

### **European Union: MDR EU 2017/745**

*Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.*

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