NEOTECH[®]
Products LLC

NeoShades[®] with Tabs Phototherapy Eye Shields

REF Catalog No.		
N720 Micro	N724 Micro	
N721 Small	N725 Small	
N722 Large	N726 Large	

Manufactured by
NEOTECH PRODUCTS LLC
28430 Witherspoon Parkway , Valencia, CA 91355, USA
1-800-966-0500 • neotechproducts.com

EMERGO EUROPE	
EC REF Prinsessegracht 20	
2514 AP, The Hague, The Netherlands	

MD Medical Device	
CE	

PPE Regulation (EU) 2016/425

Notified Body: BSI Group (2797)
Say Building, John M. Keynesplein 9
1066 EP Amsterdam, Netherlands

English
NeoShades[®] with Tabs Phototherapy Eye Shields
INDICATIONS FOR USE
The NeoShades are indicated for use as eye protection during phototherapy sessions. They are intended for use on pediatric (neonates, infants and children) patients.
DIRECTIONS FOR USE
Preparation
Step 1>Select proper size.
Approximate sizing:
Micro – 28 weeks and under
Small – 29-36 weeks
Large – 37 weeks and up
Step 2Clean and dry skin on temples per hospital protocol.
Note: It is important for skin to be as clean and dry as possible. Hydrocolloid adhesive tabs will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.
DO NOT USE ALCOHOL.

Application
Step 1Carefully remove hydrocolloid tabs from eye shields. Before removing plastic liners, hydrocolloid adhesive tabs must be actively warmed. Warming techniques include: hold between the palms of your hands for at least 60 seconds, hold near a radiant warmer for 10-15 seconds, or hold against a chemical heat warmer for 10-15 seconds.
Step 2 Peel and discard plastic liners. Place hydrocolloid adhesive tabs on temples and hold in place for 60 seconds to ensure proper adhesion.
Step 3 Cover eyes with NeoShades and gently attach to hydrocolloid adhesive tabs.
Note: Leave a portion of the tab exposed for easier removal.
Removal
Step 1 Replace NeoShades hydrocolloid adhesive tabs on skin every 5 days or per hospital protocol, whichever is sooner.
Step 2 Hold edge of hydrocolloid adhesive tab in place and slowly separate NeoShades from tab.
Step 3 Saturate hydrocolloid adhesive tab with water or saline.
Step 4 Slowly peel hydrocolloid adhesive tab away from skin as you swab with water or saline.

Repeat for opposite side.
CONTRAINDICATIONS, CAUTIONS & WARNINGS*
The NeoShades with Tabs are contraindicated for patients with facial deformation, skin infection or excoriated skin areas that would prevent the proper placement of the tabs. They are not intended for use on adults.
Federal law restricts this device to sale by or on the order of a physician.

NeoShades should only be used while the patient is under the continuous, direct supervision of healthcare professionals. When using NeoShades intermittently, always assess the integrity of the hydrocolloid tabs whenever the eye shields are removed. If the hydrocolloid tabs are pulling away from the skin, remove per the directions for use and apply new hydrocolloid tabs before resuming phototherapy.

Hydrocolloid adhesive tabs will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion. Do not use alcohol to clean skin prior to application. Discontinue use immediately if skin irritation occurs. Single patient use only.

General transportation and storage practices are applicable. Declaration of conformity can be viewed on the Neotech website: www.neotechproducts.com
Product expiration date is printed on the immediate packaging as well as the dispenser box of the product.
European Union: MDR EU 2017/745
Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.

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