# NEOTECH<sup>®</sup>

Products LLC

## NeoShades<sup>®</sup> with Tabs

### Phototherapy Eye Shields

**REF** Catalog No.

N720 Micro  
N721 Small  
N722 Large  
N724 Micro  
N725 Small  
N726 Large

 Manufactured by  
NEOTECH PRODUCTS LLC  
28430 Witherspoon Parkway, Valencia, CA 91355, USA  
1-800-966-0500 • neotechproducts.com

**EC REP** EMERGO EUROPE  
Prinsessegracht 20  
2514 AP, The Hague, The Netherlands

**MD** Medical Device

**CE**

Made in USA

D457 Rev K

PPE Regulation (EU) 2016/425

**Notified Body: BSI Group (2797),**  
Say Building, John M. Keynesplein 9  
1066 EP Amsterdam, Netherlands.

English  
NeoShades<sup>®</sup> with Tabs Phototherapy Eye Shields  
INDICATIONS FOR USE  
The NeoShades are indicated for use as eye protection during phototherapy sessions. They are intended for use on pediatric (neonates, infants and children) patients.  
DIRECTIONS FOR USE  
Preparation  
Step 1 Select proper size.  
Step 2 Clean and dry skin per hospital protocol.  
Note: It is important for skin to be as clean and dry as possible. Tabs will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.  
DO NOT USE ALCOHOL.

Application  
Step 1 Before removing liners, carefully remove tabs from NeoShades and warm tabs in hands for 60 seconds. May also be held under radiant or heel warmer for 10 to 15 seconds.  
Step 2 Peel and discard liners. Place tabs on skin and hold in place for 60 seconds to ensure proper adhesion.  
Step 3 Cover eyes with NeoShades and gently attach to tabs.  
Note: Leave a portion of the tab exposed for easier removal.  
Removal  
Step 1 Replace NeoShades tabs every 5 days or per hospital protocol, whichever is sooner.  
Step 2 Hold edge of tab in place and slowly separate NeoShades from tab.  
Step 3 Saturate tab with water or saline.  
Step 4 Slowly peel tab away from skin as you swab with water or saline.  
Repeat for opposite side.  
CONTRAINDICATIONS, CAUTIONS & WARNINGS  
The NeoShades with Tabs are contraindicated for patients with facial deformation, skin infection or excoriated skin areas that would prevent the proper placement of the tabs. They are not intended for use on adults.  
Federal law restricts this device to sale by or on the order of a physician.

NeoShades should only be used while the patient is under the continuous, direct supervision of healthcare professionals.  
Tabs should always be removed when NeoShades are taken off the patient. Do not reuse.  
Discontinue immediately if skin irritation occurs.  
Single patient use only.  
General transportation and storage practices are applicable.  
Declaration of conformity can be viewed on the Neotech website: www.neotechproducts.com  
Product expiration date is printed on the immediate packaging as well as the dispenser box of the product.  
European Union: MDR EU 2017/745  
Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.

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