**NEOTECH<sup>®</sup>**  
**Products LLC**

## NeoShades<sup>®</sup> with Headstrap Phototherapy Eye Shields

<b>REF</b>	Catalog No.
	N727 Micro
	N728 Small
	N729 Large

Manufactured by  
**NEOTECH PRODUCTS LLC**  
28430 Witherspoon Parkway , Valencia, CA 91355, USA  
1-800-966-0500 • neotechproducts.com

**EMERGO EUROPE**  
**EC REP** Westervoortsedijk 60  
6827 AT, Arnhem, The Netherlands

**MD** Medical Device

PPE Regulation (EU) 2016/425

Notified Body: BSI Group (2797)  
Say Building, John M. Keynesplein 9  
1066 EP Amsterdam, Netherlands

English  
NeoShades<sup>®</sup> with Headstrap Phototherapy Eye Shields  
**INDICATIONS FOR USE**

The NeoShades are indicated for use as eye protection during phototherapy sessions. They are intended for use on pediatric (neonates, infants and children) patients.

**DIRECTIONS FOR USE**

Step 1 Select proper size.

Approximate sizing:

Micro – 28 weeks and under

Small – 29-36 weeks

Large – 37 weeks and up

Step 2 Place headstrap behind head with smooth side of foam against head.

Step 3 Cover eyes with NeoShades.

Step 4 Gently attach hook fabric on end of headstrap to NeoShades.

Repeat for opposite side.

Note: Straps can be trimmed for proper fit.

**CAUTIONS & WARNINGS**

The NeoShades are not intended for use on adults.

Federal law restricts this device to sale by or on the order of a physician.

NeoShades should only be used while the patient is under the continuous, direct supervision of healthcare professionals.

Single patient use only.

General transportation and storage practices are applicable.

Declaration of conformity can be viewed on the Neotech website: www.neotechproducts.com

Product expiration date is printed on the immediate packaging as well as the dispenser box of the product.

European Union: MDR EU 2017/745

Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.

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