## NeoShades® with Headstrap Phototherapy Eye Shields

**[REF]** Catalog No.

N727 Micro  
N728 Small  
N729 Large

**DIRECTIONS FOR USE**  
**INSTRUCCIONES D'UTILISACION**  
**ISTRUZIONI PER L'USO**  
**INSTRUCCIONES DE USO**  
**INSTRUÇÕES DE UTILIZAÇÃO**  
**GEBRAUCHSANWEISUNG**  
**NÁVOD K POUŽITÍ**  
**ZALECENIA DOTYCZĄCE**  
**UŻYTKOWANIA**  
**ANVÄNDARINSTRUKTIONER**  
**KÄYTTÖOHJEET**  
**إرشادات الاستخدام**

Manufactured by  
NEOTECH PRODUCTS LLC  
28430 Witherspoon Parkway, Valencia, CA 91355, USA  
1-800-966-0500 • neotechproducts.com

**EC** **REP** EMERGO EUROPE  
Prinsessegracht 20  
2514 AP, The Hague, The Netherlands

**MD** Medical Device

**CE**

Made in USA

D458 Rev H

PPE Regulation (EU) 2016/425

**Notified Body: BSI Group (2797),**  
Say Building, John M. Keynesplein 9  
1066 EP Amsterdam, Netherlands.

*English*

**NeoShades® with Headstrap  
Phototherapy Eye Shields**

### INDICATIONS FOR USE

The NeoShades are indicated for use as eye protection during phototherapy sessions. They are intended for use on pediatric (neonates, infants and children) patients.

### DIRECTIONS FOR USE

**Step 1** Select proper size.

**Step 2** Place headstrap behind head with smooth side of foam against head.

**Step 3** Cover eyes with NeoShades.

**Step 4** Gently attach hook fabric on end of headstrap to NeoShades.

Repeat for opposite side.

Note: Straps can be trimmed for proper fit.

### CAUTIONS & WARNINGS

The NeoShades are not intended for use on adults.

Federal law restricts this device to sale by or on the order of a physician.

NeoShades should only be used while the patient is under the continuous, direct supervision of healthcare professionals.

Single patient use only.

General transportation and storage practices are applicable.

Declaration of conformity can be viewed on the Neotech website: www.neotechproducts.com

Product expiration date is printed on the immediate packaging as well as the dispenser box of the product.

### European Union: MDR EU 2017/745

*Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.*

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