Neotech ChinStrap™ Respiratory Support Accessory

[REF] Catalog No.	DIRECTIONS FOR USE
N784 Nano	INSTRUCCIONES D'UTILISATION
N785 Micro	ISTRUZIONI PER L'USO
N786 Small	INSTRUCCIONES DE USO
N787 Large	INSTRUÇÕES DE UTILIZAÇÃO
N788 X-Large	GEBRAUCHSANWEISUNG
N789 Macro	NÁVOD K POUŽITÍ
N790 Jumbo	ZALECENIA DOTYCZĄCE UŻYTKOWANIA
	ANVÁNDARINSTRUKTIONER
	KÄYTTÖOHJEET
	إرشادات الاستخدام

[EC] **[REP]** EMERGO EUROPE
Prinsessegracht 20
2514 AP, The Hague, The Netherlands

[MD]	Medical Device
-------------	----------------

D459 Rev H

English

Neotech ChinStrap™ Respiratory Support Accessory

INDICATIONS FOR USE

The Neotech ChinStrap is intended to keep a patient’s mouth closed during respiratory therapy. It is intended for use on pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE

Step 1 Select proper size by measuring circumference from center of chin, over the ear, to posterior of head.

Step 2 Place Neotech ChinStrap across chin with indentation facing neck and smooth, white side against skin.

Step 3 Position wider, lower portion of split chin strap under chin and upper portion across front of chin.

Step 4 Pull strap toward posterior of head with ears positioned inside ear openings and gently fasten hook tab to strap. Side marked “LEFT” should be under patient’s left ear.

Note: May be trimmed for proper fit. Ensure that strap is not too tight or pressing on ears. Hook tab should completely overlap strap.

Step 5 Replace every 5 days or per hospital protocol, whichever is sooner.

Replace immediately if worn or visibly soiled.

CAUTIONS & WARNINGS

The Neotech ChinStrap is not intended for use on adults.

Federal law restricts this device to sale by or on the order of a physician.

Discontinue immediately if skin irritation occurs.

Single patient use only.

European Union: MDR EU 2017/745

Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.

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