NEOTECH[®]
Products LLC

NeoBar[®] ET Tube Holder

REF Catalog No.	
N709 Mini	N713 Large
N710 Ultra	N714 XL
N711 Micro	N715 Macro
N712 Small	N716 Jumbo

Manufactured by
NEOTECH PRODUCTS LLC
28430 Witherspoon Parkway , Valencia, CA 91355, USA
1-800-966-0500 • neotechproducts.com

EMERGO EUROPE
Prinseesgracht 20
2514 AP, The Hague, The Netherlands

MD Medical Device	
CE	

D460 Rev J

English
NeoBar[®] ET Tube Holder
INDICATIONS FOR USE

The NeoBar is indicated for use to secure an endotracheal tube. It is intended for use on pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE
Preparation
Step 1 After patient is intubated, select proper size NeoBar by using NeoBar measuring tape strip provided.

Place mid line of strip at nasal septum and wrap to ear. Color that falls over opening of ear canal corresponds to correct NeoBar size.

Note: if unsure between two sizes, try smaller size first.
Step 2 To ensure proper fit, position NeoBar across center of mouth, between upper and lower lips. NeoBar should not contact lips. Hydrocolloid tabs must be just in front of ear.
Step 3 Clean and dry skin on cheeks per hospital protocol.
Note: It is important for skin to be as clean and dry as possible. Hydrocolloid adhesive tabs will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.
DO NOT USE ALCOHOL.

Application
Step 1 Before removing plastic liners, hydrocolloid adhesive tabs must be actively warmed. Warming techniques include: hold between the palms of your hands for at least 60 seconds, hold near a radiant warmer for 10-15 seconds, or hold against a chemical heat warmer for 10-15 seconds.
Step 2 Peel and discard plastic liners. Apply hydrocolloid adhesive tabs in front of ear on bone and hold in place for 60 seconds to ensure proper adhesion.
Note: Ensure ET tube is below platform to reduce pressure on palate or gums.
Step 3 Wrap 1/2’’ (12.5 mm) white or cloth tape completely around NeoBar platform first, then continue taping around both platform and ET tube.

Removal
Step 1 Replace NeoBar every 5 days or per hospital protocol, whichever is sooner.
Step 2 Unwrap tape to separate ET tube from NeoBar platform.
Step 3 Saturate tabs with water or saline.
Step 4 Slowly peel hydrocolloid adhesive tabs away from skin as you swab with water or saline.

CONTRAINDICATIONS, CAUTIONS & WARNINGS
The NeoBar is contraindicated for patients with infected or excoriated skin areas that would prevent the proper placement of the tabs.
It is not intended for use on adults.
Federal law restricts this device to sale by or on the order of a physician.
NeoBar should only be used while the patient is under the continuous, direct supervision of trained healthcare professionals. Discontinue use immediately if skin irritation occurs.
NeoBar should not contact lips.
Hydrocolloid adhesive tabs will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.
Do not use alcohol to clean skin prior to application.
Single patient use only.

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