NEOTECH[®] Products LLC

NeoBar® ET Tube Holder

REF Catalog No.

N709 Mini	N713 Large
N710 Ultra	N714 XL
N711 Micro	N715 Macro
N712 Small	N716 Jumbo

Manufactured by NEOTECH PRODUCTS LLC 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

EMERGO EUROPE Westervoortseidijk 60 6827 AT, Arnhem, The Netherlands

MD Medical Device

CE

Made in USA

D460 Rev L 2024.11.15

English NeoBar® ET Tube Holder INDICATIONS FOR USE The NeoBar is indicated for use to secure an endotracheal tube. It is intended for use on pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE Preparation Step 1 After patient is intubated, select proper size NeoBar by using NeoBar measuring tape strip provided.

Place mid line of strip at nasal septum and wrap to ear. Color that falls over opening of ear canal corresponds to correct NeoBar size. Note: if unsure between two sizes, try smaller size first.

Step 2 To ensure proper fit, position NeoBar across center of mouth, between upper and lower lips. NeoBar should not contact lips. Hydrocolloid tabs must be just in front of ear.

Step 3 Clean and dry skin on cheeks per hospital protocol. Note: It is important for skin to be as clean and dry as possible. Hydrocolloid adhesive tabs will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion. DO NOT USE ALCOHOL.

Application Step 1 Before removing plastic liners, hydrocolloid adhesive tabs must be actively warmed. Warming techniques include: hold between the palms of your hands for at least 60 seconds, hold near a radiant warmer for 10-15 seconds, or hold against a chemical heat warmer for 10-15 seconds.

Step 2 Peel and discard plastic liners. Apply hydrocolloid adhesive tabs in front of ear on bone and hold in place for 60 seconds to ensure proper adhesion.

Note: Ensure ET tube is below platform to reduce pressure on palate or gums. Step 3 Wrap 1/2”” (12.5 mm) white or cloth tape completely around NeoBar platform first, then continue taping around both platform and ET tube.

Removal Step 1 Replace NeoBar every 5 days or per hospital protocol, whichever is sooner. Step 2 Unwrap tape to separate ET tube from NeoBar platform. Step 3 Saturate tabs with water or saline. Step 4 Slowly peel hydrocolloid adhesive tabs away from skin as you swab with water or saline.

CONTRAINDICATIONS, CAUTIONS & WARNINGS The NeoBar is contraindicated for patients with infected or excoriating skin areas that would prevent the proper placement of the tabs. It is not intended for use on adults. Federal law restricts this device to sale by or on the order of a physician.

NeoBar should only be used while the patient is under the continuous, direct supervision of trained healthcare professionals. Discontinue use immediately if skin irritation occurs. NeoBar should not contact lips.

Hydrocolloid adhesive tabs will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.

Do not use alcohol to clean skin prior to application. Single patient use only. European Union: MDR EU 2017/745 Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.

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