
**NEOTECH**  
Products LLC

Part #	D461
Revision	K
Description	NeoGrip/VersaGrip DFU
Artwork #	AW4461
Material # (PM, DM, LM) & description	DM466
Effective date	
CO #	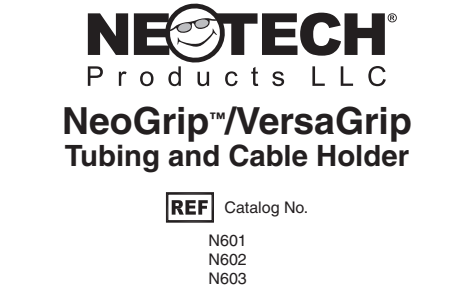

 Manufactured by  
NEOTECH PRODUCTS LLC  
28430 Witherspoon Parkway , Valencia, CA 91355, USA  
1-800-966-0500 • neotechproducts.com

 EMERGO EUROPE  
Westervoortsedijk 60  
6827 AT, Arnhem, The Netherlands

 Medical Device

Made in USA

D461 Rev K 2025.02.10

English

NeoGrip®/VersaGrip Tubing and Cable Holder

INDICATIONS FOR USE

The NeoGrip/VersaGrip is intended to organize tubes, cables and circuits.

DIRECTIONS FOR USE

Step 1 Separate hook and loop material.

Step 2 Insert tubes, cables and circuits.

Step 3 Align hook and loop and apply pressure to secure.

Step 4 To remove, slowly peel hook and loop apart and carefully remove tubes, cables, and circuits.

Note: Replace immediately if hook and loop is worn.

CAUTIONS

Do not use if hook or loop fabric is worn. Single patient use only.

General transportation and storage practices are applicable.

Declaration of conformity can be viewed on the Neotech website:

www.neotechproducts.com

Product expiration date is printed on the immediate packaging as well as the dispenser box of the product.

European Union: MDR EU 2017/745

Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.

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