## Neotech Meconium Aspirator Meconium Removal Device

|                                     |  |
|-------------------------------------|--|
| <b><span>REF</span></b> Catalog No. | <b>DIRECTIONS FOR USE</b>  |
| N0101<br>N0102                      | <b>INSTRUCTIONS D'UTILISATION</b><br><b>ISTRUZIONI PER L'USO</b><br><b>INSTRUCCIONES DE USO</b><br><b>INSTRUÇÕES DE UTILIZAÇÃO</b><br><b>GEBRAUCHSANWEISUNG</b><br><b>NÁVOD K POUŽITÍ</b><br><b>ZALECENIA DOTYCZAÇE</b><br><b>UŻYTKOWANIA</b><br><b>ANVÄNDARINSTRUKTIONER</b><br><b>KÄYTTÖOHJEET</b><br><b>إرشادات الاستخدام</b> |

 Manufactured by NEOTECH PRODUCTS LLC  
28430 Witherspoon Parkway, Valencia, CA 91355, USA  
1-800-966-0500 • neotechproducts.com

 EMERGO EUROPE  
Prinsessegracht 20  
2514 AP, The Hague, The Netherlands

 Medical Device

**Made in USA**

 D463 Rev F

*English*  
**Neotech Meconium Aspirator**  
**Meconium Removal Device**

### INDICATIONS FOR USE

The Neotech Meconium Aspirator is intended for use as a suction device to aspirate meconium from a newborn’s upper airway.

### DIRECTIONS FOR USE

**Step 1** Connect barbed end of Neotech Meconium Aspirator to suction line.

**Step 2** Set the suction pressure at 80mm Hg or less.

**Step 3** Intubate patient with proper ET tube.

**Step 4** Connect 15mm I.D. end of Neotech Meconium Aspirator to 15mm O.D. ET tube adapter.

**Step 5** Block thumb port with thumb to begin suctioning.

**Step 6** Suction for two seconds or less at a time until meconium is full removed. Suction meconium intermittently.

Note: For intermittent suctioning, unblock thumb port to stop suctioning. Block thumb port again to continue suctioning.

**Step 7** Continue to suction while ET tube is withdrawn.

**Step 8** Discard after use.

### CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

Single patient use only.

### European Union: MDR EU 2017/745

*Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.*

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