NEOTECH[®] Products LLC

Neotech Meconium Aspirator Meconium Removal Device

REF	Catalog No.
	N0101
	N0102

Manufactured by NEOTECH PRODUCTS LLC 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

EMERGO EUROPE Prinsessegracht 20 2514 AP, The Hague, The Netherlands

MD Medical Device

Made in USA

D463 Rev G

English
Neotech Meconium Aspirator Meconium Removal Device
INDICATIONS FOR USE
The Neotech Meconium Aspirator is intended for use as a suction device to aspirate meconium from a newborn's upper airway.
DIRECTIONS FOR USE
Step 1 Connect barbed end of Neotech Meconium Aspirator to suction line.
Step 2 Set the suction pressure at 80mm Hg or less.
Step 3 Intubate patient with proper ET tube.
Step 4 Connect 15mm I.D. end of Neotech Meconium Aspirator to 15mm O.D. ET tube adapter.
Step 5 Block thumb port with thumb to begin suctioning.
Step 6 Suction for two seconds or less at a time until meconium is full removed. Suction meconium intermittently.
Note: For intermittent suctioning, unblock thumb port to stop suctioning. Block thumb port again to continue suctioning.
Step 7 Continue to suction while ET tube is withdrawn.
Step 8 Discard after use.
CAUTIONS & WARNINGS
Federal law restricts this device to sale by or on the order of a physician.
Single patient use only.
European Union: MDR EU 2017/745
Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.

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