NEOTECH[®]
Products LLC

NeoPulse™ Disposable Pulse Oximeter Wrap

REF	Catalog No.
	N781 Small N782 Large N783 XL
	Manufactured by NEOTECH PRODUCTS LLC 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com
EC REP	EMERGO EUROPE Westervoortsedijk 60 6827 AT, Arnhem, The Netherlands
MD	Medical Device
	 Made in USA
	D478 Rev K 2025.03.31

English
NeoPulse™ Disposable Pulse Oximeter Wrap
INDICATIONS FOR USE
The NeoPulse is intended to hold a pulse oximeter sensor in place. It is intended for use on pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE
Step 1 Place pulse oximeter sensor per hospital protocol.
Step 2 Wrap NeoPulse around foot and pulse oximeter sensor and gently fasten hook tab to wrap.
Note: May be trimmed for proper fit. Ensure that wrap is not too tight. Hook tab should completely overlap wrap.
Step 3 Check foot after wrapping to ensure adequate circulation.
Step 4 Change the NeoPulse every 5 days or per hospital protocol, whichever is sooner.

Replace immediately if visibly worn or soiled.
CAUTIONS & WARNINGS
Discontinue use immediately if skin irritation occurs.
Single patient use only.
European Union: MDR EU 2017/745
Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.

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