**NEOTECH**<sup>®</sup>  
Products LLC

## NeoSmile™ Temperature Probe Cover

**REF** Catalog No.

N731

**DIRECTIONS FOR USE**  
**INSTRUCTIONS D'UTILISATION**  
**ISTRUZIONI PER L'USO**  
**INSTRUCCIONES DE USO**  
**INSTRUÇÕES DE UTILIZAÇÃO**  
**GEBRAUCHSANWEISUNG**  
**NÁVOD K POUŽITÍ**  
**ZALECENIA DOTYCZĄCE**  
**UŻYTKOWANIA**  
**ANVÄNDARINSTRUKTIONER**  
**KÄYTTÖOHJEET**  
**إرشادات الاستخدام**

 Manufactured by  
NEOTECH PRODUCTS LLC  
28430 Witherspoon Parkway, Valencia, CA 91355, USA  
1-800-966-0500 • neotechproducts.com

**EC** **REP** EMERGO EUROPE  
Prinsessegracht 20  
2514 AP, The Hague, The Netherlands

**MD** Medical Device

Made in USA

D480 Rev F

*English*  
**NeoSmile™**  
Temperature Probe Cover

### INDICATIONS FOR USE

The NeoSmile is intended to cover and secure a temperature probe.

### DIRECTIONS FOR USE

#### Preparation

Clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. NeoSmile will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

#### Application

**Step 1** Before removing liner, warm NeoSmile in hands for 60 seconds. May also be held under radiant or heel warmer for 10 to 15 seconds

**Step 2** Peel and discard liner. Place NeoSmile over probe and skin and hold in place for 60 seconds to ensure proper adhesion.

#### Removal

**Step 1** Replace NeoSmile every 5 days or per hospital protocol, whichever is sooner.

**Step 2** Saturate NeoSmile with water or saline.

**Step 3** Slowly peel NeoSmile away from skin as you swab with water or saline.

### CAUTIONS & WARNINGS

Discontinue immediately if skin irritation occurs.

NeoSmile will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Do not reuse or reapply.

Single patient use only.

### European Union: MDR EU 2017/745

*Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.*

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