NEOTECH [®] Products LLC	
NeoSmile™ Temperature Probe Cover	
	
REF Catalog No. N731	
	
Manufactured by NEOTECH PRODUCTS LLC 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com	
EC REF	EMERGO EUROPE Prinsessegracht 20 2514 AP, The Hague, The Netherlands
MD	Medical Device
	
CE	 Made in USA
PPE Regulation (EU) 2016/425	D480 Rev G
Notified Body: BSI Group (2797) Say Building, John M. Keynesplein 9 1066 EP Amsterdam, Netherlands	

English
NeoSmile™ Temperature Probe Cover
INDICATIONS FOR USE
The NeoSmile is intended to cover and secure a temperature probe.
DIRECTIONS FOR USE
Preparation
Clean and dry skin where NeoSmile will be placed per hospital protocol.
Note: It is important for skin to be as clean and dry as possible. NeoSmile will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.
DO NOT USE ALCOHOL.
Application
Step 1 Before removing plastic liner, hydrocolloid adhesive must be actively warmed. Warming techniques include: hold between the palms of your hands for at least 60 seconds, hold near a radiant warmer for 10-15 seconds, or hold against a chemical heel warmer for 10-15 seconds.
Step 2 Peel and discard liner. Place NeoSmile over probe and skin and hold in place for 60 seconds to ensure proper adhesion.
Removal
Step 1 Replace NeoSmile every 5 days or per hospital protocol, whichever is sooner.
Step 2 Saturate NeoSmile with water or saline.
Step 3 Slowly peel NeoSmile away from skin as you swab the skin with water or saline.
CAUTIONS & WARNINGS
Discontinue immediately if skin irritation occurs.
NeoSmile will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.
Do not use alcohol to clean skin prior to application.
Do not reuse or reapply.
Single patient use only.
European Union: MDR EU 2017/745
Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.

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