NEOTECH®

Products LLC

EZ-Hold®

Tubing and Cannula Holder

[REF] Catalog No.	DIRECTIONS FOR USE
N756	INSTRUCTIONS D'UTILISATION
N757	ISTRUZIONI PER L'USO
N758	INSTRUCCIONES DE USO
	INSTRUÇÕES DE UTILIZAÇÃO
	GEBRAUCHSANWEISUNG
	NÁVOD K POUŽITÍ
	ZALECENIA DOTYCZAÇE
	UŻYTKOWANIA
	ANVÄNDARINSTRUKTIONER
	KÄYTTÖOHJEET
	إرشادات الاستخدام

Manufactured by
NEOTECH PRODUCTS LLC
28430 Witherspoon Parkway, Valencia, CA 91355, USA
1-800-966-0500 • neotechproducts.com

EMERGO EUROPE
Prinsessegracht 20
2514 AP, The Hague, The Netherlands

Medical Device

D483 Rev G

English
EZ-Hold®
Tubing and Cannula Holder

INDICATIONS FOR USE

The EZ-Hold is intended to anchor and support breathing tubes.

DIRECTIONS FOR USE

Preparation

Clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. EZ-Hold will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

Application

Step 1 Before removing liners, warm EZ-Hold in hands for 60 seconds. May also be held under radiant or heel warmer for 10 to 15 seconds.

Step 2 Discard white, paper liner, then peel and discard clear, plastic liner from hydrocolloid base

Step 3 Apply base to skin and hold in place for 60 seconds to ensure proper adhesion.

Step 4 Peel and discard second liner and fold flap over tubing.

Note: Lift flap as needed to reposition or remove tubing.

Removal

Step 1 Replace EZ-Hold every 5 days or per hospital protocol, whichever is sooner.

Step 2 Lift flap to remove tubing.

Step 3 Saturate EZ-Hold with water or saline.

Step 4 Slowly peel EZ-Hold away from skin as you swab with water or saline.

CAUTIONS & WARNINGS

Discontinue immediately if skin irritation occurs.

EZ-Hold will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Single patient use only.

European Union: MDR EU 2017/745

Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.

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