## EZ-Hold®

### Tubing and Cannula Holder

**REF** Catalog No.

N756  
N757  
N758

Manufactured by  
NEOTECH PRODUCTS LLC  
28430 Witherspoon Parkway , Valencia, CA 91355, USA  
1-800-966-0500 • neotechproducts.com

EMERGO EUROPE  
EC REP  
Westervoortsedijk 60  
6827 AT, Arnhem, The Netherlands

**MD** Medical Device

**CE**

Made in USA

D483 Rev J

English

EZ-Hold® Tubing and Cannula Holder

#### INDICATIONS FOR USE

The EZ-Hold is intended to anchor and support breathing tubes.

#### DIRECTIONS FOR USE

Preparation

Clean and dry skin where EZ-Hold will be placed per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. EZ-Hold will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

Application

Step 1 Before removing plastic liner, hydrocolloid adhesive must be actively warmed. Warming techniques include: hold between the palms of your hands for at least 60 seconds, hold near a radiant warmer for 10-15 seconds, or hold against a chemical heel warmer for 10-15 seconds.

Step 2 Discard white, paper liner, then peel and discard clear, printed liner from hydrocolloid base

Step 3 Apply base to skin and hold in place for 60 seconds to ensure proper adhesion.

Step 4 Peel and discard second frosted liner and fold flap over tubing.

Note: Lift flap as needed to reposition or remove tubing.

Removal

Step 1 Replace EZ-Hold every 5 days or per hospital protocol, whichever is sooner.

Step 2 Lift flap to remove tubing.

Step 3 Saturate EZ-Hold with water or saline.

Step 4 Slowly peel EZ-Hold away from skin as you swab the skin with water or saline.

#### CAUTIONS & WARNINGS

Discontinue immediately if skin irritation occurs.

EZ-Hold will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Single patient use only.

European Union: MDR EU 2017/745

Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.

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