NEOTECH®

Products LLC

NeoLead®

ECG Electrodes

[REF] Catalog No.	DIRECTIONS FOR USE
N300	INSTRUCTIONS D'UTILISATION
N301	ISTRUZIONI PER L'USO
N305	INSTRUCCIONES DE USO
N306	INSTRUÇÕES DE UTILIZAÇÃO
N315	GEBRAUCHSANWEISUNG
	NÁVOD K POUŽITÍ
	ZALECENIA DOTYCZĄCE
	UŻYTKOWANIA
	ANVÄNDARINSTRUKTIONER
	KÄYTTÖOHJEET
	إرشادات الاستخدام

 Manufactured by NEOTECH PRODUCTS LLC 28430 Witherspoon Parkway, Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

EC	REP	EMERGO EUROPE Prinsessegracht 20 2514 AP, The Hague, The Netherlands
-----------	------------	--

MD	Medical Device
-----------	----------------

D484 Rev H

NeoLead ECG Electrodes should only be used while patient is under continuous direct supervision of healthcare professionals.

Electrodes will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Do not use NeoLead (N300, N301) during x-ray or MRI procedures.

Do not use Radiolucent NeoLead (N305, N306, N315) during MRI procedures.

Discontinue immediately if skin irritation occurs.

Disposable. Do not reuse.

Single patient use only.

© 2012 Neotech Products LLC

English
NeoLead®
ECG Electrodes

© 2012 Neotech Products LLC

INDICATIONS FOR USE

The NeoLead® ECG Electrodes are intended for ECG monitoring. They are intended for use on pediatric (neonates, infants and children) patients.

© 2012 Neotech Products LLC

DIRECTIONS FOR USE

Preparation

Clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. Electrodes will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

Application

Step 1 Before removing liner, warm the lead in hands for 60 seconds. May also be held under radiant warmer or heel warmer for 10 to 15 seconds.

Step 2 Peel and discard liner.

Step 3 Place the lead on skin and hold in place for 60 seconds to ensure proper adhesion.

Removal

Step 1 Replace NeoLead every 5 days or per hospital protocol, whichever is sooner.

Step 2 Saturate the lead with water or saline.

Step 3 Slowly peel the lead away from the skin as you swab with water or saline.

© 2012 Neotech Products LLC

CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

NeoLead ECG Electrodes should only be used while patient is under continuous direct supervision of healthcare professionals.

Electrodes will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Do not use NeoLead (N300, N301) during x-ray or MRI procedures.

Do not use Radiolucent NeoLead (N305, N306, N315) during MRI procedures.

Discontinue immediately if skin irritation occurs.

Disposable. Do not reuse.

Single patient use only.

© 2012 Neotech Products LLC

European Union: MDR EU 2017/745

Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.

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