NeoBond® Hydrocolloid Adhesive Strips

REF Catalog No.

N741 Pre-cut Strips
N742 Roll

Manufactured by
NEOTECH PRODUCTS LLC
28430 Witherspoon Parkway , Valencia, CA 91355, USA
1-800-966-0500 • neotechproducts.com

EMERGO EUROPE
Westervoortsedijk 60
6827 AT, Arnhem, The Netherlands

MD Medical Device

Made in USA

D490 Rev H

English
NeoBond® Hydrocolloid Adhesive Strips

INDICATIONS FOR USE
NeoBond Hydrocolloid Adhesive Strips are intended for general use as a medical adhesive.

DIRECTIONS FOR USE

Preparation
Clean and dry skin where NeoBond will be placed per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. NeoBond will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

Application

Step 1 Before removing from plastic liner, hydrocolloid adhesive must be actively warmed. Warming techniques include: hold between the palms of your hands for at least 60 seconds, hold near a radiant warmer for 10-15 seconds, or hold against a chemical heat warmer for 10-15 seconds.

Step 2 Peel from liner. Wrap or apply NeoBond over skin and product (e.g. feeding tube, limb board) and hold in place for 60 seconds to ensure proper adhesion. Discard liner when done."

Removal

Step 1 Replace NeoBond every 5 days or per hospital protocol, whichever is sooner.

Step 2 Saturate the NeoBond with water or saline.

Step 3 Slowly peel NeoBond away from skin as you swab the skin with water or saline.

CAUTIONS & WARNINGS

Discontinue immediately if skin irritation occurs.

NeoBond will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Single patient use only.

European Union: MDR EU 2017/745

Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.

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