NEOTECH[®]
Products LLC

NeoSeal[®] Nasal Prong Seal

REF Catalog No.

N420 Micro
N421 Small
N422 Large

Manufactured by
NEOTECH PRODUCTS LLC
28430 Witherspoon Parkway, Valencia, CA 91355, USA
1-800-966-0500 • neotechproducts.com

EMERGO EUROPE
Prinsessegracht 20
2514 AP, The Hague, The Netherlands

Medical Device

Made in USA

D420 Rev G

English

NeoSeal[®] Nasal Prong Seal
INDICATIONS FOR USE

The NeoSeal is intended to create a seal around the nares and protect the skin of the septum and nares. It is intended for use on pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE

Step 1 Peel NeoSeal from liner.

Step 2 Slip over nasal prongs with adhesive facing cannula.

Note: Soft, smooth side without adhesive should contact patient's skin.

Step 3 Place cannula per hospital protocol.

Step 4 Replace NeoSeal daily or per hospital protocol, whichever is sooner.

Note: Replace immediately if soiled or wet.

CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

Choking hazard. NeoSeal should only be used while the patient is under the continuous, direct supervision of trained healthcare professionals.

Replace immediately if soiled or wet.

Discontinue immediately if skin irritation occurs.

Single patient use only.

European Union: MDR EU 2017/745

Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.

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