**NEOTECH**<sup>®</sup>  
Products LLC

## NeoSeal<sup>®</sup> Nasal Prong Seal

<b>REF</b>	Catalog No.
<span></span>	N420 Micro
<span></span>	N421 Small
<span></span>	N422 Large

Manufactured by NEOTECH PRODUCTS LLC 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

EMERGO EUROPE Westervoortsedijk 60 6827 AT, Arnhem, The Netherlands

**MD** Medical Device

English NeoSeal<sup>®</sup> Nasal Prongs Seal INDICATIONS FOR USE The NeoSeal is intended to create a seal around the nares and protect the skin of the nasal septum and nares. It is intended for use on pediatric (neonates, infants and children) patients. DIRECTIONS FOR USE Step 1 Peel NeoSeal from plastic liner. Step 2 Slip over nasal prongs with adhesive facing cannula. Note: Soft, smooth side without adhesive should contact patient’s skin. Step 3 Insert prongs per hospital protocol. Document placement of the NeoSeal in the patient’s medical record. Note: To minimize risk of aspiration/ingestion, ensure the nasal cannula prongs and/or NeoSeal do not enter the infant’s mouth. Step 4 Replace NeoSeal daily or per hospital protocol, whichever is sooner. Note: Replace immediately if soiled or wet. Step 5 Document removal and/or replacement in the patient’s medical record. CAUTIONS & WARNINGS Federal law restricts this device to sale by or on the order of a physician. Choking hazard. NeoSeal should only be used while the patient is under the continuous, direct supervision of trained healthcare professionals. Risk of ingestion or aspiration. When in use, ensure the nasal cannula prongs and/or NeoSeal do not enter the infant’s mouth. Replace immediately if soiled or wet. Discontinue use immediately if skin irritation occurs. Single patient use only. European Union: MDR EU 2017/7745

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