SoftSil™


Silicon Adhesive Patch

REF Catalog No. N745

Manufactured by
NEOTECH PRODUCTS LLC
28430 Witherspoon Parkway , Valencia, CA 91355, USA
1-800-966-0500 • neotechproducts.com

EMERGO EUROPE
Westervoortsedijk 60
6827 AT, Arnhem, The Netherlands

MD Medical Device

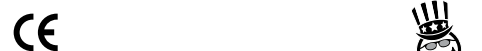D493 Rev G

English
SoftSil™ Silicon Adhesive Patch
INDICATIONS FOR USE
SoftSil Silicon Adhesive Patch intended for general use as a medical adhesive.
DIRECTIONS FOR USE
Step 1 Clean and dry skin per hospital protocol.
Note: It is important for skin to be as clean and dry as possible. SoftSil will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.
DO NOT USE ALCOHOL.
Step 2 Remove from liner and apply to skin. Reposition as needed.
Note: Can be cut to size.
Step 3 Replace SoftSil every 2 days or per hospital protocol, whichever is sooner.
Replace immediately if visibly soiled.
CAUTIONS & WARNINGS
Do not use to secure life-sustaining devices.
Discontinue immediately if skin irritation occurs.
SoftSil will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.
Do not use alcohol to clean skin prior to application.
Single patient use only.
European Union: MDR EU 2017/745
Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.

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