SoftSil™ Silicon Adhesive Patch

REF Catalog No. N745

DIRECTIONS FOR USE
ISTRUZIONI PER L'USO
INSTRUCCIONES DE USO
INSTRUÇÕES DE UTILIZAÇÃO
GEBRAUCHSANWEISUNG
NÁVOD K POUŽITÍ
ZALECENIA DOTYCZĄCE
UŻYTKOWANIA
ANVÄNDARINSTRUKTIONER
KÄYTTÖOHJEET
إرشادات الاستخدام

Manufactured by
NEOTECH PRODUCTS LLC
28430 Witherspoon Parkway, Valencia, CA 91355, USA
1-800-966-0500 • neotechproducts.com

EC REP EMERGO EUROPE
Prinsessegracht 20
2514 AP, The Hague, The Netherlands

MD Medical Device

CE

Made in USA

D493 Rev E

English
SoftSil™
Silicon Adhesive Patch

INDICATIONS FOR USE

SoftSil Silicon Adhesive Patch intended for general use as a medical adhesive.

DIRECTIONS FOR USE

Step 1 Clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. SoftSil will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

Step 2 Remove from liner and apply to skin. Reposition as needed.

Note: Can be cut to size.

Step 3 Replace SoftSil every 2 days or per hospital protocol, whichever is sooner.

Replace immediately if visibly soiled.

CAUTIONS & WARNINGS

Do not use to secure life-sustaining devices.

Discontinue immediately if skin irritation occurs.

SoftSil will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Single patient use only.

European Union: MDR EU 2017/745

Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.

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