Hold-A-Line™ Catheter and Tubing Holder

REF] Catalog No.	DIRECTIONS FOR USE
N705	INSTRUCTIONS D’UTILISATION
	ISTRUZIONI PER L’USO
	INSTRUCCIONES DE USO
	INSTRUÇÕES DE UTILIZAÇÃO
	GEBRAUCHSANWEISUNG
	NÁVOD K POUŽITÍ
	ZALECENIA DOTYCZĄCE
	UŻYTKOWANIA
	ANVÄNDARINSTRUKTIONER
	KÄYTTÖOHJEET
	إرشادات الاستخدام

EC REP	EMERGO EUROPE Prinsessegracht 20 2514 AP, The Hague, The Netherlands
----------------------	--

MD	Medical Device
-----------	----------------

D494 Rev F

English
Hold-A-Line™
Catheter and Tubing Holder

INDICATIONS FOR USE

The Hold-A-Line is intended to anchor and support umbilical catheters and tubing. It is intended for use on pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE

Preparation

After catheter or tube is in place, clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. Hold-A-Line will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

Application

Step 1 Before removing liners, warm Hold-A-Line in hands for 60 seconds. May also be held under radiant or heel warmer for 10 to 15 seconds.

Step 2 Discard white, paper liner, then peel and discard clear, plastic liner from hydrocolloid base.

Step 3 Apply base to skin and hold in place for 60 seconds to ensure proper adhesion.

Step 4 Peel and discard frosted liner and adhere catheter or tube to base.

Step 5 Fold flap over catheter or tube, align with base and press firmly together, enclosing catheter or tube between adhesives.

Step 6 Lift flap as needed to reposition or remove catheter or tube.

Removal

Step 1 Replace Hold-A-Line every 5 days or per hospital protocol, whichever is sooner.

Step 2 Lift flap to remove catheter or tube.

Step 3 Saturate Hold-A-Line with water or saline.

Step 4 Slowly peel Hold-A-Line away from skin as you swab with water or saline.

CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

Hold-A-Line should only be used while the patient is under the continuous, direct supervision of trained healthcare professionals.

Discontinue immediately if skin irritation occurs.

Hold-A-Line will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Single patient use only.

European Union: MDR EU 2017/745

Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.

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