



Manufactured by **NEOTECH PRODUCTS LLC** 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

EMERGO EUROPE Westervoortsedijk 60 6827 AT, Arnhem, The Netherlands

MD Medical Device



CE **REF** Catalog No. N705

English **NEOTECH PRODUCTS LLC** 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

EMERGO EUROPE Westervoortsedijk 60 6827 AT, Arnhem, The Netherlands

MD Medical Device

CE **REF** Catalog No. N705

English **NEOTECH PRODUCTS LLC** 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

EMERGO EUROPE Westervoortsedijk 60 6827 AT, Arnhem, The Netherlands

MD Medical Device

CE **REF** Catalog No. N705

English **NEOTECH PRODUCTS LLC** 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

EMERGO EUROPE Westervoortsedijk 60 6827 AT, Arnhem, The Netherlands

MD Medical Device


CE **REF** Catalog No. N705

English **NEOTECH PRODUCTS LLC** 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

EMERGO EUROPE Westervoortsedijk 60 6827 AT, Arnhem, The Netherlands

MD Medical Device

CE **REF** Catalog No. N705

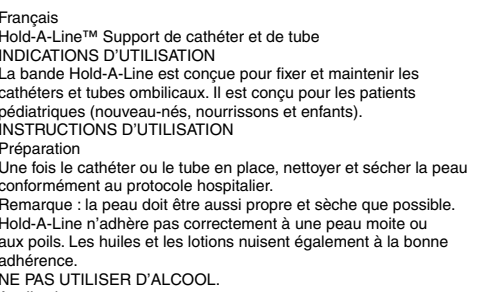

Français **NEOTECH PRODUCTS LLC** 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

EMERGO EUROPE Westervoortsedijk 60 6827 AT, Arnhem, The Netherlands

MD Medical Device

CE **REF** Catalog No. N705

English **NEOTECH PRODUCTS LLC** 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

EMERGO EUROPE Westervoortsedijk 60 6827 AT, Arnhem, The Netherlands

MD Medical Device

CE **REF** Catalog No. N705

Italiano **NEOTECH PRODUCTS LLC** 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

EMERGO EUROPE Westervoortsedijk 60 6827 AT, Arnhem, The Netherlands

MD Medical Device

CE **REF** Catalog No. N705

English **NEOTECH PRODUCTS LLC** 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

EMERGO EUROPE Westervoortsedijk 60 6827 AT, Arnhem, The Netherlands

MD Medical Device

CE **REF** Catalog No. N705

Español **NEOTECH PRODUCTS LLC** 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

EMERGO EUROPE Westervoortsedijk 60 6827 AT, Arnhem, The Netherlands

MD Medical Device

CE **REF** Catalog No. N705

Português **NEOTECH PRODUCTS LLC** 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

EMERGO EUROPE Westervoortsedijk 60 6827 AT, Arnhem, The Netherlands

MD Medical Device

CE **REF** Catalog No. N705

English **NEOTECH PRODUCTS LLC** 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

EMERGO EUROPE Westervoortsedijk 60 6827 AT, Arnhem, The Netherlands

MD Medical Device

CE **REF** Catalog No. N705

English **NEOTECH PRODUCTS LLC** 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

EMERGO EUROPE Westervoortsedijk 60 6827 AT, Arnhem, The Netherlands

MD Medical Device

CE **REF** Catalog No. N705

Deutsch **NEOTECH PRODUCTS LLC** 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

EMERGO EUROPE Westervoortsedijk 60 6827 AT, Arnhem, The Netherlands

MD Medical Device

CE **REF** Catalog No. N705

Česky **NEOTECH PRODUCTS LLC** 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

EMERGO EUROPE Westervoortsedijk 60 6827 AT, Arnhem, The Netherlands

MD Medical Device

CE **REF** Catalog No. N705

Poliski **NEOTECH PRODUCTS LLC** 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

utrudniając prawidłowo mocowanie produktu.

NIE WOLNO UŻYWAĆ ALKOHOLU.

Zakładanie
Krok 1 Klej hydrokoloidowy musi być aktywnie ogrzewany przed usunięciem plastikowej osłonki. Techniki rozgrzewania obejmują: trzymanie między dłońmi przez co najmniej 60 sekund, trzymanie w pobliżu ogrzewacza promiennikowego przez 10-15 sekund lub trzymanie przy chemicznym ogrzewaczu pięt przez 10-15 sekund.
Krok 2 Usuń białą papierową warstwę ochronną, a następnie usuń przezroczystą plastykową folię z przylepnej warstwy hydrokoloidowej.

Krok 3 Umieść przylepną warstwę plastra na skórze i przytrzymaj przez 60 sekund, aby umożliwić prawidłowe przymocowanie.
Krok 4 Usuń przezroczystą warstwę ochronną i przymocuj cewnik lub przewód do warstwy przylepnej.
Krok 5 Zawrń skrzydełko na cewniku lub przewodzie, ustaw je zgodnie z warstwą przylepną i zdecydowanie docisnij, mocując cewnik lub przewód między warstwami przylepnymi.
Krok 6 Podnieś skrzydełko, jeżeli konieczne jest przesunięcie lub usunięcie cewnika lub przewodu.

Usuwanie
Krok 1 Wymieniaj plaster Hold-A-Line na nowy co 5 dni lub częściej, zależnie od procedur szpitalnych.
Krok 2 Podnieś skrzydełko, aby usunąć cewnik lub przewód.
Krok 3 Nasącz plaster Hold-A-Line wodą lub solą fizjologiczną.
Krok 4 Powoli odklej plaster Hold-A-Line od skóry, używając waćka nasączonego wodą lub solą fizjologiczną.
PRZESTROGI I OSTRZEŻENIAW
Przepisy federalne zezwalają na sprzedaż tego produktu wyłącznie przez lekarzy lub na ich zlecenie.
Plaster Hold-A-Line powinien być używany wyłącznie w przypadku pacjentów pozostających pod stałym nadzorem przeszkolonego personelu medycznego.
W przypadku wystąpienia podrażnienia skóry należy zrezygnować z użycywnia produktu.
Na wilgotnej lub owosionej skórze plaster Hold-A-Line nie zostanie prawidłowo zamocowany. Oleje i płyny kosmetyczne również utrudniają prawidłowe mocowanie produktu.
Nie wolno czyszczyć skóry alkoholem przed przymocowaniem produktu.
Produkt jest przeznaczony do użyciu tylko dla pojedynczego pacjenta.
Unia Europejska: MDR EU 20177745
Kazdy nowy incydent zwiazany z tym wyrobem nalezy zglosic spólcze Neotech Products i właściwemu organowi państwa członkowskiego, w którym użytkownik lub pacjent maja miejsce zamieszkania.

Svenska **NEOTECH PRODUCTS LLC** 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

EMERGO EUROPE Westervoortsedijk 60 6827 AT, Arnhem, The Netherlands

MD Medical Device

CE **REF** Catalog No. N705

Suomi **NEOTECH PRODUCTS LLC** 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

EMERGO EUROPE Westervoortsedijk 60 6827 AT, Arnhem, The Netherlands

MD Medical Device

CE **REF** Catalog No. N705

toimivaltaille viranomaiselle, johon käytettävä ja/tai potilas on sijoituttanut.

Latviešu **NEOTECH PRODUCTS LLC** 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

EMERGO EUROPE Westervoortsedijk 60 6827 AT, Arnhem, The Netherlands

MD Medical Device

CE **REF** Catalog No. N705

Български **NEOTECH PRODUCTS LLC** 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

EMERGO EUROPE Westervoortsedijk 60 6827 AT, Arnhem, The Netherlands

MD Medical Device

CE **REF** Catalog No. N705

Hrvatski **NEOTECH PRODUCTS LLC** 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com

EMERGO EUROPE Westervoortsedijk 60 6827 AT, Arnhem, The Netherlands

