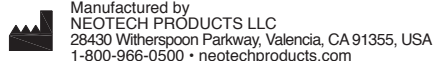
**NEOTECH®**  
Products LLC

## NeoBridge® Umbilical Catheter Holder

<b>[REF]</b> Catalog No.	<b>DIRECTIONS FOR USE</b> <b>INSTRUCTIONS D'UTILISATION</b> <b>ISTRUZIONI PER L'USO</b> <b>INSTRUCCIONES DE USO</b> <b>INSTRUÇÕES DE UTILIZAÇÃO</b> <b>GEBRAUCHSANWEISUNG</b> <b>NÁVOD K POUŽÍTÍ</b> <b>ZALECENIA DOTYCZĄCE</b> <b>UZÝTKOWANIA</b> <b>ANVÄNDARINSTRUKTIONER</b> <b>KÄYTTÖOHJEET</b> <b>إرشادات الاستخدام</b>
N700 Micro N701 Small N702 Large	
	
<b>[EC]</b> <b>[REF]</b> EMERGO EUROPE Prinsessegracht 20 2514 AP, The Hague, The Netherlands	

**[EC]** **[REF]** EMERGO EUROPE Prinsessegracht 20 2514 AP, The Hague, The Netherlands

**[EC]** **[REF]** EMERGO EUROPE Prinsessegracht 20 2514 AP, The Hague, The Netherlands

**[MD]** Medical Device

**[MD]** Medical Device

D495 Rev E

*English*  
**NeoBridge®**  
**Umbilical Catheter Holder**

#### INDICATIONS FOR USE

The NeoBridge is intended to anchor and support umbilical catheters. It is intended for use on pediatric (neonates, infants and children) patients.

#### DIRECTIONS FOR USE

##### Preparation

After catheter is has been inserted, clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. NeoBridge will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

##### Application

**Step 1** Before removing liners, warm NeoBridge in hands for 60 seconds. May also be held under radiant or heel warmer for 10 to 15 seconds.

**Step 2** Discard white, paper liner, then peel and discard clear, plastic liner from hydrocolloid base.

**Step 3** With slit on right side and wider end of hydrocolloid base facing patient’s head, slip catheter through center of NeoBridge.

**Step 4** Apply base to skin and hold in place for 60 seconds to ensure proper adhesion.

**Step 5** Peel and discard frosted liner from top half of flap. Bring flap up and affix catheter(s) to flap in an arch or U shape to minimize tension with catheter(s) exiting out the side of the flap.

Note: Do not extend catheter(s) into green area of flap. If using two catheters, have one catheter exit each side of flap.

**Step 6** Peel and discard frosted liner from bottom half of flap. Bring the flap up to align green area of bottom half with green area of top half and press firmly together, enclosing catheter(s) between adhesive flaps.

**Step 7** Pull flaps apart to remove or reposition catheter(s) as needed, then close to secure.

##### Removal

**Step 1** Replace NeoBridge every 5 days or per hospital protocol, whichever is sooner

**Step 2** Pull flaps apart to remove catheter(s).

**Step 3** Saturate NeoBridge with water or saline.

**Step 4** Slowly peel NeoBridge away from skin as you swab with water or saline.

#### CAUTIONS & WARNINGS

Federal law restricts this device to sale by or on the order of a physician.

NeoBridge should only be used while the patient is under the continuous, direct supervision of trained healthcare professionals.

Discontinue immediately if skin irritation occurs.

NeoBridge will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Single patient use only.

**European Union: MDR EU 2017/745**

*Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.*

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