<span></span>	
<b>NEOTECH</b> <sup>®</sup> Products LLC	
<b>NeoBridge</b> <sup>®</sup> Umbilical Catheter Holder	
<span></span>	
<b>REF</b> Catalog No.	
N700 Micro N701 Small N702 Large	
<span></span>	
Manufactured by NEOTECH PRODUCTS LLC 28430 Witherspoon Parkway , Valencia, CA 91355, USA 1-800-966-0500 • neotechproducts.com	
<b>EMERGO EUROPE</b> Westervoortsedijk 60 6827 AT, Arnhem, The Netherlands	
<b>MD</b> Medical Device	
<span></span>	
<b>CE</b>	<b> Made in USA</b>
<span></span>	
D495 Rev H 2025.03.10	
<span></span>	

English
NeoBridge® Umbilical Catheter Holder
INDICATIONS FOR USE
The NeoBridge is intended to anchor and support umbilical catheters. It is intended for use on pediatric (neonates, infants and children) patients.
DIRECTIONS FOR USE
Preparation
After catheter is has been inserted, clean and dry skin per hospital protocol.
Note: It is important for skin to be as clean and dry as possible. NeoBridge will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.
DO NOT USE ALCOHOL.

Application
Step 1 Before removing plastic liner, hydrocolloid adhesive must be actively warmed. Warming techniques include: hold between the palms of your hands for at least 60 seconds, hold near a radiant warmer for 10-15 seconds, or hold against a chemical heat warmer for 10-15 seconds.
Step 2 Discard from white, paper liner, then peel and discard clear, plastic liner from hydrocolloid base.
Step 3 With slit on right side and wider end of hydrocolloid base facing patient’s head, slip catheter through center of NeoBridge.
Step 4 Apply base to skin and hold in place for 60 seconds to ensure proper adhesion.
Step 5 Peel and discard frosted liner from top half of flap. Bring flap up and affix catheter(s) to flap in an arch or U shape to minimize tension with catheter(s) exiting out the side of the flap.
Note: Do not extend catheter(s) into green area of flap. If using two catheters, have one catheter exit each side of flap.
Step 6 Peel and discard frosted liner from bottom half of flap. Bring the flap up to align green area of bottom half with green area of top half and press firmly together, enclosing catheter(s) between adhesive flaps.
Step 7 Flip flaps apart to remove or reposition catheter(s) as needed, then close to secure.

Removal
Step 1 Replace NeoBridge every 5 days or per hospital protocol, whichever is sooner.
Step 2 Pull flaps apart to remove catheter(s).
Step 3 Saturate NeoBridge with water or saline.
Step 4 Slowly peel NeoBridge away from skin as you swab the skin with water or saline.
CAUTIONS & WARNINGS
Federal law restricts this device to sale by or on the order of a physician.

NeoBridge should only be used while the patient is under the continuous, direct supervision of trained healthcare professionals. Discontinue immediately if skin irritation occurs. NeoBridge will not adhere properly to moist skin or hair. Oils and lotions will adversely affect adhesion. Do not use alcohol to clean skin prior to application. Single patient use only.
European Union: MDR EU 2017/745
Any serious incident that has occurred in relation to this device should be reported to Neotech Products and the competent authority of the Member State in which the user and/or patient is established.

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